

# 5900

## Comfort Care, Elite Recliner



*PLEASE READ AND FAMILIARIZE YOURSELF WITH ALL INSTRUCTIONS BEFORE USING THIS PRODUCT. If you have trouble understanding these instructions contact your dealer or Winco customer support, (800) 237-3377 before attempting to use this product; otherwise injury may occur.*

***Winco assumes no responsibility for damage or injury caused by the improper assembly, installation or use of these products; or during assembly or maintenance of these products.***



### ***TO REDUCE THE RISK OF INJURY TO PERSONS:***

1. READ AND FOLLOW ALL DIRECTIONS.
2. NEVER use side table as a seat.
3. DO NOT put hands, feet or clothing into any openings when changing positions on recliner. Attendant MUST confirm that users hands and feet are safe while changing recliner positions or **SERIOUS INJURY MAY RESULT**.
4. Stay clear of the recline mechanisms.
5. Periodically, recheck tightness of all nuts, bolts and screws.
6. Never use the chair arms or backrest as a seat; **SERIOUS INJURY MAY OCCUR**.
7. Lock casters at all times, except when transporting chair.
8. Chair must **ALWAYS** be positioned on a level surface.
9. **DO NOT** use recliner for Transporting in or with ANY type of vehicle or trailer. Winco recliners have not been tested or approved for use by an occupant in any type of vehicle or trailer.
10. Immediately remove from service; Any recliner with broken recline mechanisms, torn upholstery, or other mechanical or visible damage.
11. **USE ONLY WINCO AUTHORIZED REPLACEMENT PARTS.**
12. NEVER EXCEED the recommended **weight capacity of 275 lbs.** (124.7 kg).
13. **SAVE THESE INSTRUCTIONS** for future reference and training.

***DO NOT*** LIFT OR CHANGE CHAIR POSITION BY USING THE LEGREST.  
*THIS COULD CAUSE INJURY TO USER OR DAMAGE TO THE RECLINE MECHANISM AND WILL VOID WARRANTY.*  
***ALWAYS CHANGE POSITION FROM BACK OF CHAIR.***

**5900 Weight Capacity = 275 lbs. (124.7 kg)**


### **Assembly Instructions - NO TOOLS REQUIRED**

*PLEASE READ AND FAMILIARIZE YOURSELF WITH ALL INSTRUCTIONS BEFORE PROCEEDING WITH ASSEMBLY*

#### **Install the Back:**

1. After checking your product for any shipping damage, chair needs to be positioned upright on level surface. Remove all packing material and hardware that was secured for shipping. Remove tape and plastic. If using scissors or knife, please be careful not to damage upholstery.
2. Make sure footrest is in closed position & **LOCK ALL CASTERS** before installing chair back.

## REAR of Chair

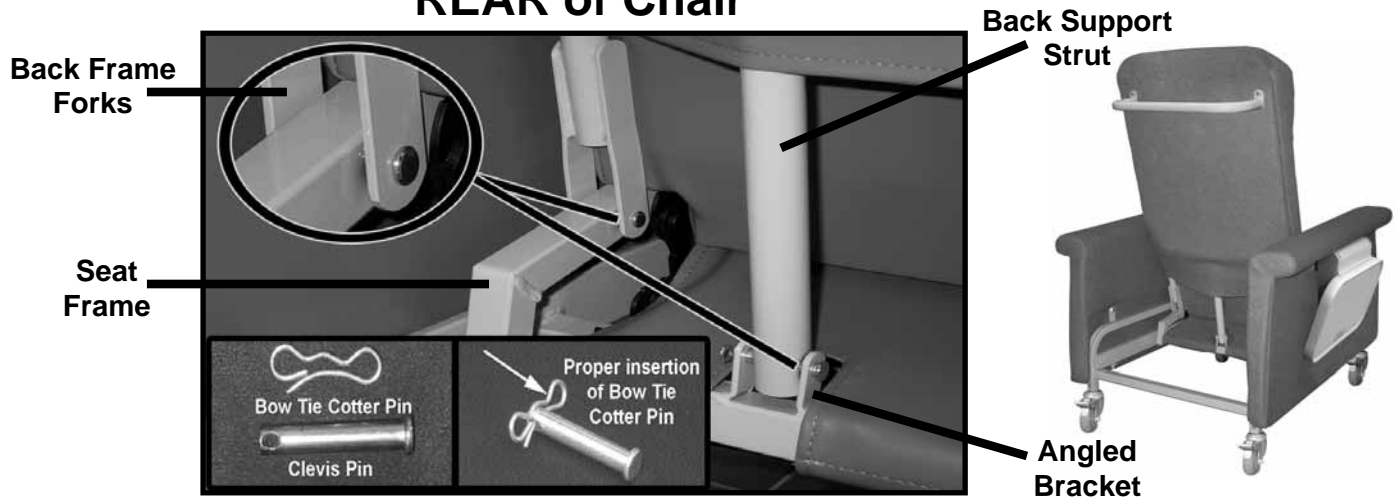

3. Remove the (2) **Bow Tie Cotter Pins** and the (2) **Clevis Pins** from the lower **Back Frame Forks**. **DO NOT LOOSE THESE PARTS, YOU WILL NEED THEM AGAIN!**
4. Using a second person to help; Position the **Back Frame Forks** over the **Seat Frame** and align Backframe Fork holes with holes in Seat Frame Rail.

**HINT:** If you do not have a helper; Rest the bottom of the Upholstered Back on the rear portion of the seat and push the Back Assembly slightly forward to help steady the chair back.

5. Use (1) one of the **Clevis Pins** (with tapered end) and insert it THROUGH the aligned holes on **Back Frame Fork and Seat Rail Tube**. REPEAT FOR OPPOSITE SIDE OF CHAIR. (see above *Clevis Pin Install photo*)



Failure to correctly install clevis pins connecting Back Frame fork holes and Seat Frame Rail holes, will cause back to operate poorly and may cause injury to the user or damage the chair and void warranty.

6. Insert (2) two **Bow Tie Cotter Pins**, into the small holes on ends of the (2) two **Clevis Pins** you just installed.
7. Remove **Bow Tie Cotter Pin** and **Clevis Pin** from the loose end of the **Back Support Strut** (Strut is attached to center of backframe).
8. Align the holes on the loose end of the **Back Support Strut** with the holes in the **Angle Brackets** (on chair frame) and insert the Clevis Pin into these aligned holes and secure with the **Bow Tie Cotter Pin**.


### Operation of the Chair:

9. This recliner has three (3) positions; UPRIGHT, SEMI-RECLINE, and RECLINE, which can be achieved by user or attendant.
10. This chair is equipped with **Total Lock casters**. Stepping on the brake tab will lock the wheel and the swivel mechanism of the caster for added safety and security.

NOTE: Optional Rigid Casters do not lock.

To ensure proper operation of the reclining mechanism, lubricate all pivot points with a silicone type lubricant (or lubricant approved by your facility) at least twice yearly. Periodically recheck tightness of all screws and bolts.

**Wrinkles in Vinyl** – Storage and shipping can cause wrinkles of the vinyl on your Winco chair. This is normal. Please allow 1-2 weeks at room temperature (approx. 70 degrees) to allow for vinyl to expand back to original appearance

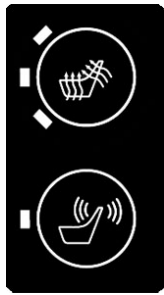**HEAT****MASSAGE****-OR-****HEAT ONLY****HEAT & MASSAGE OPTIONS - (Models with Heat-HT or Heat & Massage-HM)**

Control is located on the inside of right arm of chair.

**Heating:** Push **HEAT** button – See indicator light on switch

**3 Lights** = High Heat

**2 Lights** = Medium Heat

**1 Lights** = Low Heat

**No Light** = HEAT OFF

Allow 5-8 minutes for back and seat to warm to desired TEMP. setting.

Heat will automatically shut off after approximately 60 minutes.

*(Timer resets when switch is pressed)*

**Massage (optional):** Push **MASSAGE** button

**1X** = Lower Back / Low Intensity

**2X** = Lower Back / High Intensity

**3X** = Upper & Lower Back / Low Intensity

**4X** = Upper & Lower Back / High Intensity

**5X** = MASSAGE OFF


Massage will automatically shut off after approximately 15 minutes. *(Timer resets when switch is pressed)*

- Use a DAMP CLOTH ONLY to clean switch, wiring or power pack (located under seat frame).
- If there is no heat or massage or flashing light on switch - check power cord connections to wall outlet AND connection of power cord to power pack unit under chair.
- If switch displays a flashing light; Check all accessible wire connectors under chair for possible loose connections, then contact Winco customer service: 1-800-237-3377.

**CORD STORAGE**

*(Models with Heat-HT or Heat & Massage-HM)*

1. Before transporting chair, or when heat & massage is not in use, power cord should be coiled and stored in the "Hook & Loop" fastener provided as shown, to prevent tripping on cord, damage to cord, damage to recliner, etc.

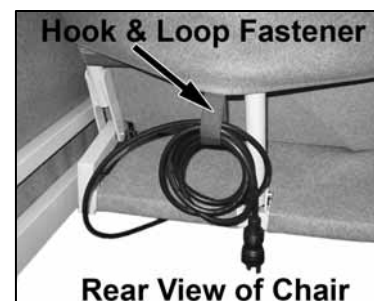

**NOTE:**  
**Keep cord CLEAR of any moving parts**



**FOR PRODUCTS WITH POWERED OPTIONS (Heat, Massage, etc...)**

***Read all instructions before using this product.***

**IMPORTANT SAFETY INSTRUCTIONS - SAVE THESE INSTRUCTIONS**

***To reduce the risk of electric shock when using an electrical appliance, basic precautions should always be followed, including the following:***

DANGER - Always unplug this product from the electrical outlet before cleaning, maintenance or when putting on or taking off parts.

WARNING:

1. Unplug this product from the electrical outlet if this product is not to be used for extended periods of time.
2. Supervision should be provided when this product is used by, on, or near children, invalids, or disabled persons.
3. Use this product only for its intended use as described in this manual. Do not use attachments not recommended by the manufacturer.
4. Never operate this product if it has a damaged cord or plug, if it is not working properly, if it has been dropped or damaged, or dropped into water. Return the product to an authorized Winco service center.
5. DO NOT carry or pull this product by power supply cord or use the cord as a handle.
6. Keep the cord away from heated surfaces.
7. Do not use outdoors.
8. To disconnect, turn all controls to the "off" position, then remove plug from outlet. Do not "yank" cord.
9. Clean switch with a damp cloth only – Warranty will be VOID if liquid is introduced into switch mechanism.
10. Use only a certified extension cord that is rated at 15 amperes; extension cords rated for less amperage may over heat. Care should be taken to arrange the cord so that the cord cannot be pulled or tripped over.
11. DO NOT run recliner or other equipment over cord.
12. Recliner comes with a grounded three blade plug, if the plug does not fit into the outlet; contact a certified electrician to install the proper outlet. DO NOT MODIFY THE PLUG IN ANY WAY.

DANGER:

- i) Never insert pins, or attach other metallic fasteners into/onto the upholstery of this chair.
- ii) Do not use this product if the covering shows signs of deterioration, such as checking, blistering, or cracking.
- iii) KEEP DRY - Do not operate in a wet condition or environment.

WARNING:

- i) Long term, continuous exposure to heat may cause burns. It is recommended that skin in contact with heated area be checked for redness and blistering during long term usage. Do not use on an infant, invalid or unconscious person. Do not use on insensitive skin or on a person with poor blood circulation. If you are uncertain if you should use this product, please consult your physician about the use of heat and massage.
- ii) Do Not Crush/Pinch heating elements or wiring.

WARNING: Connect this product to a properly grounded outlet only.

***GROUNDING INSTRUCTIONS:***

This product must be grounded. If it should malfunction or breakdown, grounding provides a path of least resistance for electric current to reduce the risk of electric shock. This product is equipped with a cord having an equipment-grounding conductor and a grounding plug. The plug must be plugged into an appropriate outlet that is properly installed and grounded in accordance with all local codes and ordinances.

DANGER - Improper connection of the equipment-grounding conductor can result in a risk of electric shock. Check with a qualified electrician or serviceman if you are in doubt as to whether the product is properly grounded. Do not modify the plug provided with the product - if it will not fit the outlet, have a proper outlet installed by a qualified electrician.

This product is for use on a nominal 120-volt circuit and has a grounding plug. Make sure that the product is connected to an outlet having the same configuration as the plug. No adapter should be used with this product.

For an added level of safety; Winco recommends the use of a 120 volt GFCI (ground fault circuit interrupter) outlet.

Visit our website [www.wincomfg.com/product-documents/parts-lists](http://www.wincomfg.com/product-documents/parts-lists) for a complete parts list catalog of your chair or call Customer Service @ 1-800-237-3377 to request it by mail.



**Other Quality Winco Medical Furnishings You May Be Interested In:**



Clinical Recliners




Exam Tables



Convalescent (LTC) Recliners



Ring Binder Carts

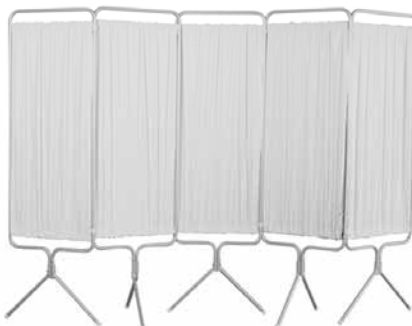

Privacy Screens



Phlebotomy Chairs



Specialty Chair Transfer Chairs



Stools



Room Chairs

Visit us online at [www.wincomfg.com](http://www.wincomfg.com)



ISO 9001-2008 CERTIFIED



## LIMITED WARRANTY

• 5516 SW 1<sup>st</sup> Lane • Ocala, FL 34474  
 • Phone (352) 854-2929 • (800) 237-3377  
 • Fax (352) 854-9544  
 • www.wincomfg.com  
 • customerservice@wincomfg.com

### TERMS

*Winco Mfg., LLC. warrants this product to be free of manufacturer's defects in material and workmanship, provided that the product is used according to normal operating conditions and proper maintenance intended by manufacturer. This warranty is available only to the original retail purchaser, is non-transferable and commences on the date of retail sale; proof of purchase required.*

**Limited One Year Complete Product Warranty:** Winco Mfg., LLC warrants the complete product for one (1) year. At Winco Mfg., LLC. sole discretion, it may repair or replace any components freight free that are found to be defective during the first year. *Winco Mfg., LLC. shall not be liable for any labor, or any other costs incurred as a result of or in conjunction with a warranty claim.*

**Limited Three Year Warranty:** Winco Mfg., LLC warrants the recliner mechanisms, electrical components, vinyl [from cracking or delaminating] pressurized gas springs, when new, for a period of three (3) years. At Winco Mfg., LLC. sole discretion, it may repair or replace components found to be defective. *Winco Mfg., LLC. shall not be liable for any labor, shipping or any other costs incurred as a result of or in conjunction with a warranty claim.*

**Limited Lifetime Chair Frame Warranty:** Winco Mfg., LLC warrants the steel chair frame, when new, for the lifetime of the original purchaser. This limited warranty does not apply to paint/finish or any components attached to the frame such as; upholstery, foam, casters, mechanisms or related parts that are covered under above warranties. *Winco Mfg., LLC shall not be liable for any labor, shipping or any other costs incurred as a result of or in conjunction with a warranty claim.*

The purchaser's exclusive remedy under this warranty shall be limited to such repair or replacement of defective components at Winco Mfg., LLC sole discretion. For warranty service, contact the dealer that the product was originally purchased from or Winco Mfg., LLC directly.

### EXCLUSIONS

There are no other warranties, conditions, representations or guarantees, express or implied, made or intended by Winco Mfg., LLC and all other warranties, conditions, representations or guarantees including any warranties, conditions, representations or guarantees under any Sale of Good Act or Like legislation or statute is hereby expressly excluded. Any and all other implied warranties shall not extend beyond the duration of the express warranty. Liability for incidental or consequential damages is excluded to the extent permitted by law. Some states do not allow the exclusion or limitation so the above limitation or exclusion may not apply to you. This warranty gives you specific legal rights. In addition you may also have other rights, which vary from state to state.

### GENERAL PROVISIONS

No warranty herein contained or set out shall apply when damage or repair is caused by any of the following:

1. Damage in transit.
2. Accident, alteration, abuse or misuse of product ie. exceeding weight capacities, applying inordinate pressure to footrest/leg rest, use in Mental Health facilities or like institutions.
3. Fire, water damage, theft, war, riot, hostility, acts of God.



*Examples contained in this list are not to be construed as all-inclusive.*