






PARTS LIST

Image Number	Note	Part Number	Description	QTY per Product
1	A	U3006XX	857 UPHOLSTERED TOP	1
1	A,B	U3401XX	857 UPHOLSTERED HEADREST	1
1	B	222801	857 TABLE FRAME	2
1	B	701010	1X2X18 RECT INSERT CAP/GLIDE LT.GRAY	4
1	B	566519	SCREW-MACH 1/4-20 X 2 1/2 PAN PHILLIP M/S ZINC	4
1	B	566528	SCREW-TAP 10 X 2 3/4 PAN PHIL T/S ZINC	4
1	B	222830	857 CYLINDER BRACKET WELDMENT	1
1	B	566504	SCREW-MACH 1/4-20 X 3/4 RND PHILLIP M/S ZINC	2
1	B	201250	857 TABLE HINGE POWDER COATED	2
1	B	566506	SCREW-MACH 1/4-20 X 1 1/2 RND PHIL M/S ZINC	8
1	B	566204	NUT-1/4-20 HEX NYLON LOCKNUT ZINC	8
1	B	204720	POWDER COATED CYLINDER HINGE BRKT	1
1	B	566511	SCREW-MACH 10-24 X 3/4 PAN PHIL M/S ZINC	2
1	B	700255	GROMMET, HINGE BLACK	2
1	B	700260	GROMMET, SLOT 12 IN, BLACK	1
2		222720	850/851/855/856 WELDED BRACE ASSM.	1
2		566401	BOLT-CARRIAGE 1/4-20 X 1 1/2 ZINC	4
2		566204	NUT-1/4-20 HEX NYLON LOCKNUT ZINC	4
3	C	222730	WELDED, 850-851-856 REINF. BRACE ASSM WELDED	1
3		566401	BOLT-CARRIAGE 1/4-20 X 1 1/2 ZINC	4
3		566204	NUT-1/4-20 HEX NYLON LOCKNUT ZINC	4
4	D	222731	WELDED, 850-851-856 REINF. BRACE ASSM WELDED	1
4		566401	BOLT-CARRIAGE 1/4-20 X 1 1/2 ZINC	4
4		566204	NUT-1/4-20 HEX NYLON LOCKNUT ZINC	4

NOTES

Note	Description
A	XX- Use color code for specific part number i.e. 02 - Royal Blue
B	These parts are included with the UPHOLSTERED TOP It is NOT recommended that these parts be sold seperately however in special cases they can be.
C	For tables with optional cabinet, or drawer. Must order double-sided tape listed on Page 3
D	For tables with optional shelf. Must order double-sided tape listed on Page 3

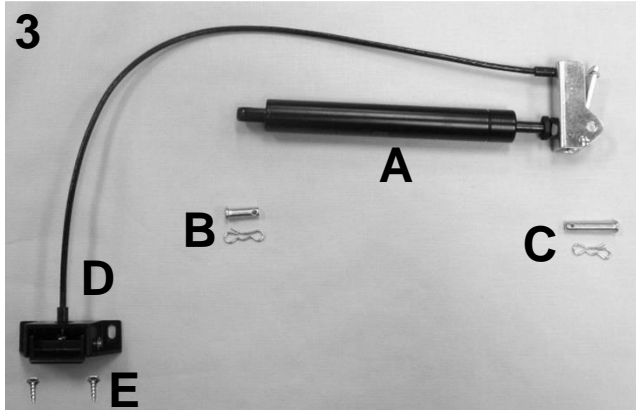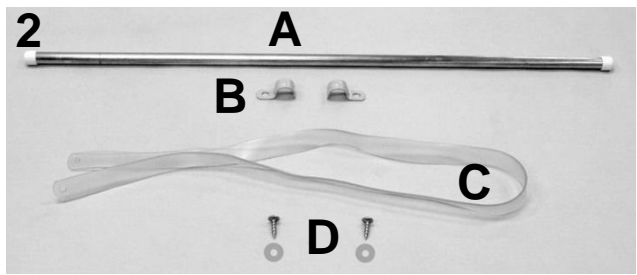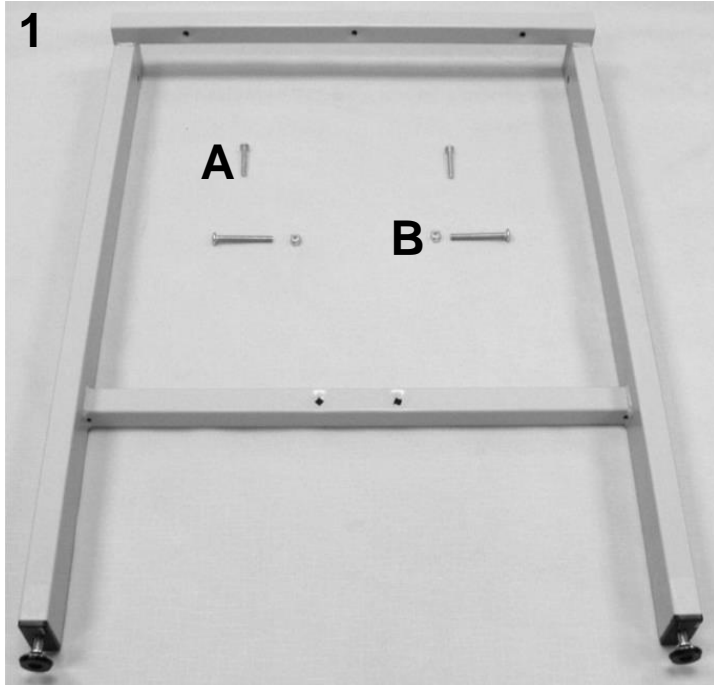

PARTS LIST

Image Number	Note	Part Number	Description	QTY per Product
1		222708K	WELDED LEG ASSM. KIT 19"	2
1	A	222708	WELDED LEG ASSM. 19"	1
1		222709K	WELDED LEG ASSM. KIT 25" (850 style)	2
1	A	222709	WELDED LEG ASSM. 25" (850 style)	1
1		222710K	WELDED LEG ASSM. KIT 30" (850 style)	2
1	A	222710	WELDED LEG ASSM. 30" (850 style)	1
1		222711K	WELDED LEG ASSM. KIT 36" (850 style)	2
1	A	222711	WELDED LEG ASSM. 36" (850 style)	1
1	A	500350	GLIDE NON-SWIVEL ZINC 5/16X1" NP LN	2
1	A	501065	1"x2" METAL GLIDE INSERT;18 GAUGE;5/16-18 THREAD	2
1A	A	566506	SCREW-MACH 1/4-20 X 1 1/2 RND PHIL M/S ZINC	2
1B	A	566519	SCREW-MACH 1/4-20 X 2 1/2 PAN PHILLIP M/S ZINC	2
1B	A	566204	NUT-1/4-20 HEX NYLON LOCKNUT ZINC	2
2		222705K	PAPER ROD HOLDER ASSM. KIT (850 STYLE)	1
2A	A	222705	850/851/855/856 ROD, PAPER HOLDER 1/2" ALUM	1
2A	A	700650	1/2" PLASTIC CAP, WHITE	2
2B		201810	PAPER ROD CLAMP, GRAY	2
2C		271500	VINYL TABLE STRAP	1
2D		566302	WASHER-#10 SAE F/W ZINC	2
2D		566534	SCREW-TAP 10 X 3/4 RND WASHER SQ T/S ZINC	2
3A		205225	857 GAS CYL. W/OPERATOR ASSM'D & ADJ'D	1
3B		506010	CLEVIS PIN, CUSTOM (.307 x 1.688)	1
3B		566002	BOW TIE COTTER PIN .072	1
3C		566009	5/16 X 1 CLEVIS PIN ZINC CP-84	1
3C		566002	BOW TIE COTTER PIN .072	1
3D	B	403450	REMOTE RELEASE SYSTEM	1
3E		566534	SCREW-TAP 10 X 3/4 RND WASHER SQ T/S ZINC	2

NOTES

Note	Description
A	These parts are included in ALL above listed kit.
B	Item is NOT included when ordering IMAGE 3A



PARTS LIST

Image Number	Note	Part Number	Description	QTY per Product
1		263960K	850 SERIES PLAIN SHELF KIT	1
1	A	263960	850 SERIES PLAIN SHELF	1
N/A	A	705600	DOUBLE SIDED FOAM TAPE, 3/4" X 36YDS	1 ft.
2		263950	850,851,856 MODULE SHELF	1
3		266780K	850/851/856 CABINET MODULE ASSM. KIT	1
3A	A	266780	850/851/856 CABINET MODULE	1
3A	A	566519	SCREW-MACH 1/4-20 X 2 1/2 PAN PHILLIP M/S ZINC	4
3A	A	566204	NUT-1/4-20 HEX NYLON LOCKNUT ZINC	4
4		266730K	850/851/856 DRAWER MODULE KIT	1
4	A	266730	850/851/856 DRAWER MODULE	1
4B	A	566519	SCREW-MACH 1/4-20 X 2 1/2 PAN PHILLIP M/S ZINC	4
4B	A	566204	NUT-1/4-20 HEX NYLON LOCKNUT ZINC	4
4A	A	266760K	850/851/856 MODULE DRAWER ASSM. KIT	1
4A	A	266760	850/851/856 MODULE DRAWER ASSM	1
4B	A	566519	SCREW-MACH 1/4-20 X 2 1/2 PAN PHILLIP M/S ZINC	4
4B	A	566204	NUT-1/4-20 HEX NYLON LOCKNUT ZINC	4

NOTES

Note	Description
A	These parts are included in the above listed kit.



Revision History

REV	Effective Date	CCR Number	Description of Change
01	11/27/17	DCC 17-071	Misc. minor changes.