

8570

## Adjustable Headrest Treatment Table with Cabinets and/or Drawer Modules



CUSTOMER INSTRUCTIONS

*PLEASE READ AND FAMILIARIZE YOURSELF WITH ALL INSTRUCTIONS BEFORE USING THIS PRODUCT. If you have trouble understanding these instructions contact your dealer or Winco customer support, (800) 237-3377 before attempting to use this product; otherwise injury may occur.*

**Winco assumes no responsibility for damage or injury caused by the improper assembly, installation or use of these products; or during assembly or maintenance of these products.**



1. READ AND FOLLOW ALL DIRECTIONS.
2. NEVER stand on table or INJURY MAY OCCUR.
3. NEVER allow user to sleep unattended on table or INJURY MAY OCCUR.
4. NEVER leave an unconscious user unattended on table or INJURY MAY OCCUR.
5. DO NOT use table for transporting user in or with ANY type of vehicle or trailer. Winco tables have not been tested or approved for use by an occupant in any type of vehicle or trailer.
6. NEVER leave an infant or child unattended on table or INJURY MAY OCCUR.
7. DO NOT put hands, feet or clothing into any openings when changing positions on table. Attendant MUST confirm that users hands and feet are safe while changing table positions or INJURY MAY RESULT.
8. Periodically, recheck tightness of all nuts, bolts and screws.
9. USE ONLY WINCO AUTHORIZED REPLACEMENT PARTS.
10. NEVER EXCEED the recommended **weight capacity of 400 lbs.** (181.4 kg).
11. SAVE THESE INSTRUCTIONS for future reference and training.

**8570 Weight Capacity = 400 lbs. (181.4 kg.)**

### **Assembly Instructions:**

#### Tools needed:

- Scissors or box cutter
- Phillips-head screwdriver
- ½" wrench or ½" socket & ratchet wrench
- 7/16" wrench, (multiple size wrench included to check pre-installed fasteners for tightness after shipping)
- - If using powered tools always start hardware together by hand first & wear safety goggles

1. After checking your product for any shipping damage, remove all packing material and hardware that was secured for shipping.
2. Use a ½" wrench or socket, and remove the four (4) bolts that are attaching the table to the skid. (Bolts go through the plastic **Threaded Inserts** in ends of table legs, and the shipping skid.)
3. Using at least (2) people, lift the table off of the skid.
4. Install the included **Adjustable Glide Feet** into the **Threaded Inserts**, located in the bottom the table legs. (see note).

**NOTE:** Turning table on side or upside down will ease this installation, however, drawers should be removed. Please protect your Upholstered Top with cardboard, clean carpeted floor, etc... when performing this operation. Use of two or more people is HIGHLY recommended.

5. With table in the upright position, adjust the (4) **Adjustable Glide Feet** to level the table.

6. Use a Phillips bit screwdriver and a 7/16" wrench to check all screws, nuts, and bolts to ensure they are tight and did not loosen during shipping.
7. Load paper roll onto **Paper Holder Rod**. Rod must be inserted through **Paper Rod Clamps**.

### Operation of Table

8. To raise the **Upholstered Backrest**, pull black lever on the **Remote Release System**. To lower the Backrest, pull the same black lever with one hand and apply pressure to the Backrest with your other hand until the desired position is achieved, then release the black lever.



***Do not over tighten nuts and bolts. Periodically check all fasteners for tightness.***

**Vinyl Strap Adjustment** – Should the vinyl strap on your Winco table need to be adjusted, unscrew the screw and washer that currently holds either side of it in place. Pierce another hole in the strap to give desired tension and re-insert screw and washer.


**Wrinkles in Vinyl** – Storage and shipping can cause wrinkles of the vinyl on your Winco table. This is normal. Please allow 1-2 weeks at room temperature (approx. 70 degrees) to allow for vinyl to expand back to original appearance.

<u>Items factory installed on table:</u>	<u>Qty.</u>	<u>Sizing chart – part number</u>
Table Frame Rails	2	222801
Paper Rod Clamp	2	501810
Cylinder Bracket Weldment	1	222830
Backrest Hinge	2	301250
Cylinder Hinge Bracket	1	404720
Vinyl Strap (for paper)	1	271500
Gas Spring(cylinder)	1	405225
Remote Release System	1	403450
Threaded Insert Caps (black)	4	500105
Upholstered Top	1	CALL FOR SPECIAL PART NUMBERS
Upholstered Backrest	1	CALL FOR SPECIAL PART NUMBERS
 <u>Optional, D-modules (drawers):</u>		
Drawer Module	1	266730
Module Drawer Assembly	2	266760
Drawer Pull	2	403630
 8500 Series Module Shelf	 1	 263950
 <u>Optional, C-modules (cabinets):</u>		
Cabinet Module	1	266780
8500 Series Module Shelf	1	263950

### Items included with table (not factory installed):

Glide Feet (leveling feet)	4	500350
Paper Holder Rod	1	222705

**8570-D1 (drawer) model shown.**  
 (similar Cabinet model not shown)

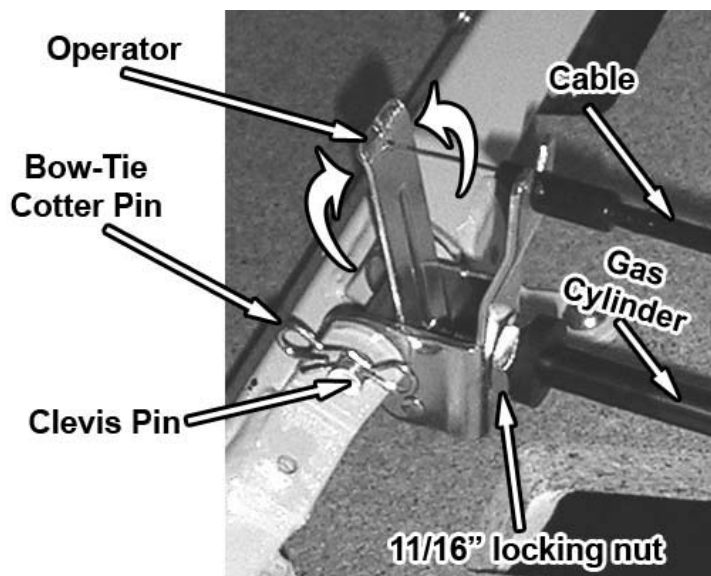

**ADJUSTING GAS SPRING (cylinder):**

1. Remove **Cable** from **Operator**.
2. Loosen **11/16"** nut on top of **Operator**.
3. Remove **Bow-Tie Cotter Pin** from **Clevis Pin**.
4. Remove **Clevis Pin**.
5. Turn the entire **Operator** one full revolution:  
 Clockwise if backrest does not move when handle is actuated  
 OR Counter-clockwise if backrest will not stay in down position.

Finish the adjustment:

6. Re-install the **Clevis Pin**.
7. Re-install the **Bow-Tie Cotter Pin**.
8. Tighten the **11/16"** locking nut on top of **Operator**.
9. Re-install **cable**.
10. REPEAT these steps as needed until proper backrest action is achieved.



## LIMITED WARRANTY

5516 SW 1<sup>st</sup> Lane • Ocala, FL 34474  
 Phone (352) 854-2929 • (800) 237-3377 • Fax (352)  
 854-9544

### TERMS

*Winco Mfg., LLC. warrants this product to be free of manufacturer's defects in material and workmanship, provided that the product is used according to normal operating conditions and proper maintenance intended by manufacturer. This warranty is available only to the original retail purchaser, is non-transferable and commences on the date of retail sale; proof of purchase required.*

**Limited One Year Complete Product Warranty:** Winco Mfg., LLC warrants the complete product for one (1) year. At Winco Mfg., LLC. sole discretion, it may repair or replace any components freight free that are found to be defective during the first year. *Winco Mfg., LLC. shall not be liable for any labor, or any other costs incurred as a result of or in conjunction with a warranty claim.*

**Limited Three Year Warranty:** Winco Mfg., LLC warrants the recliner mechanisms, electrical components, vinyl [from cracking or delaminating] pressurized gas springs, when new, for a period of three (3) years. At Winco Mfg., LLC. sole discretion, it may repair or replace components found to be defective. *Winco Mfg., LLC. shall not be liable for any labor, shipping or any other costs incurred as a result of or in conjunction with a warranty claim.*

**Limited Lifetime Chair Frame Warranty:** Winco Mfg., LLC warrants the steel chair frame, when new, for the lifetime of the original purchaser. This limited warranty does not apply to paint/finish or any components attached to the frame such as; upholstery, foam, casters, mechanisms or related parts that are covered under above warranties. *Winco Mfg., LLC shall not be liable for any labor, shipping or any other costs incurred as a result of or in conjunction with a warranty claim.*

The purchaser's exclusive remedy under this warranty shall be limited to such repair or replacement of defective components at Winco Mfg., LLC sole discretion. For warranty service, contact the dealer that the product was originally purchased from or Winco Mfg., LLC directly.

### EXCLUSIONS

There are no other warranties, conditions, representations or guarantees, express or implied, made or intended by Winco Mfg., LLC and all other warranties, conditions, representations or guarantees including any warranties, conditions, representations or guarantees under any Sale of Good Act or Like legislation or statute is hereby expressly excluded. Any and all other implied warranties shall not extend beyond the duration of the express warranty. Liability for incidental or consequential damages is excluded to the extent permitted by law. Some states do not allow the exclusion or limitation so the above limitation or exclusion may not apply to you. This warranty gives you specific legal rights. In addition you may also have other rights, which vary from state to state.

### GENERAL PROVISIONS

No warranty herein contained or set out shall apply when damage or repair is caused by any of the following:

1. Damage in transit.
2. Accident, alteration, abuse or misuse of product ie. exceeding weight capacities, applying inordinate pressure to footrest/leg rest, use in Mental Health facilities or like institutions.
3. Fire, water damage, theft, war, riot, hostility, acts of God.



*Examples contained in this list are not to be construed as all-inclusive.*