

8050 Adjusting Table

with Adjustable Facerest / Armrest



PLEASE READ AND FAMILIARIZE YOURSELF WITH ALL INSTRUCTIONS BEFORE USING THIS PRODUCT. If you have trouble understanding these instructions contact your dealer or Winco customer support, (800) 237-3377 before attempting to use this product; otherwise injury may occur.

Winco assumes no responsibility for damage or injury caused by improper assembly, installation, use, or maintenance of these products.



1. READ AND FOLLOW ALL DIRECTIONS.
2. NEVER stand on table or INJURY MAY OCCUR.
3. NEVER allow user to sleep unattended on table or INJURY MAY OCCUR.
4. NEVER leave an unconscious user unattended on table or INJURY MAY OCCUR.
5. DO NOT use table for transporting user in or with ANY type of vehicle or trailer. Winco tables have not been tested or approved for use by an occupant in any type of vehicle or trailer.
6. NEVER leave an infant or child unattended on table or INJURY MAY OCCUR.
7. DO NOT allow children to play on or with the table or INJURY MAY OCCUR.
8. MAKE SURE Headrest Adjustment Notched Bar is fully engaged before user lays on table.
9. DO NOT put hands, feet or clothing into any openings when changing positions on table. Attendant MUST confirm that users hands and feet are safe while changing table positions or INJURY MAY RESULT.
10. Periodically, recheck tightness of all nuts, bolts and screws.
11. USE ONLY WINCO AUTHORIZED REPLACEMENT PARTS.
12. NEVER EXCEED the recommended **weight capacity of 400 lbs.** (181.4 kg).
13. SAVE THESE INSTRUCTIONS for future reference and training.

8050 Weight Capacity = 400 lbs. (181.4 kg.)

Tools needed:

- Phillips-head screwdriver
- Two (2) 7/16" wrenches (multiple size wrenches included)
- Scissors or box cutter
- Cardboard shipping container (stays under table during assembly to protect upholstery)
- - If using powered tools always start hardware together by hand first -

PLEASE READ AND FAMILIARIZE YOURSELF WITH ALL INSTRUCTIONS BEFORE PROCEEDING WITH ASSEMBLY

1. After checking your product for any shipping damage, table needs to be positioned upside-down on cardboard or protective surface. Remove all packing material and hardware that was secured for shipping. Cut tape on corners of box so that cardboard will lay flat while you assemble table.

Arm Frame Assembly

2. Using two (2) 7/16" wrenches; assemble the two **Arm Frame Standoffs** to the **Arm Frame Support** using (2) 1/4-20 x 1-3/4" bolts and locknuts. See picture below.


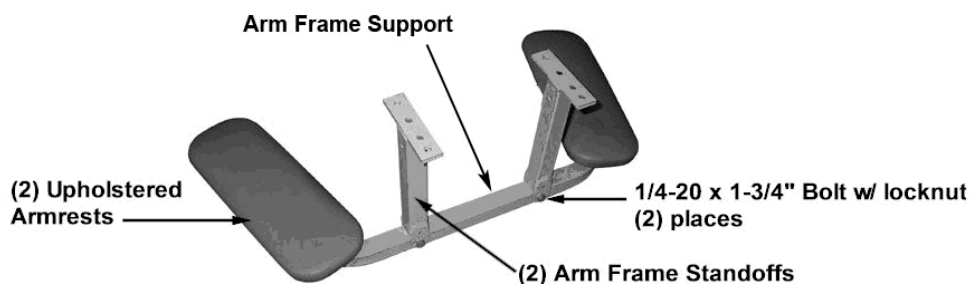


Table Components Installation

3. Position the table upside-down on the cardboard shipping container or carpeted floor to protect upholstery.
4. Install the **Face Paper Holder** on the bottom of the **Upholstered Face Rest** using two (2) “pre-installed” 1” black drywall screws that are located on the bottom of the **Upholstered Face Rest** (across from the **Face Paper Cutter**.)
Holes in the **Face Paper Holder** will be aligned with drywall screw holes. Do not over-tighten.
5. Install the complete **Arm Frame Support Assembly** (you previously assembled) onto **Uph. Face Rest** using the (4) “pre-installed” ¼-20 x 1” Phillips head screws; located in **Face Rest**.
See complete table assembly picture.
6. Use a 7/16” wrench and install the **Headrest Adjustment Assembly** to the bottom of the **Upholstered Top**, using (2) ¼-20 x 1-3/4” hex-head bolts in the (2) sets of threaded holes in the wood of the **Upholstered Top**.
7. Install the **Ratchet Bar** (end with hole) to the inside of the **LEFT Arm Frame Standoff** (Left; as if laying face-down on table), with (1) ¼-20 x 1-½” screw; (1) 1/16” nylon spacer and (1) ¼-20 locknut.
(SEE PHOTOS BELOW & NOTE)

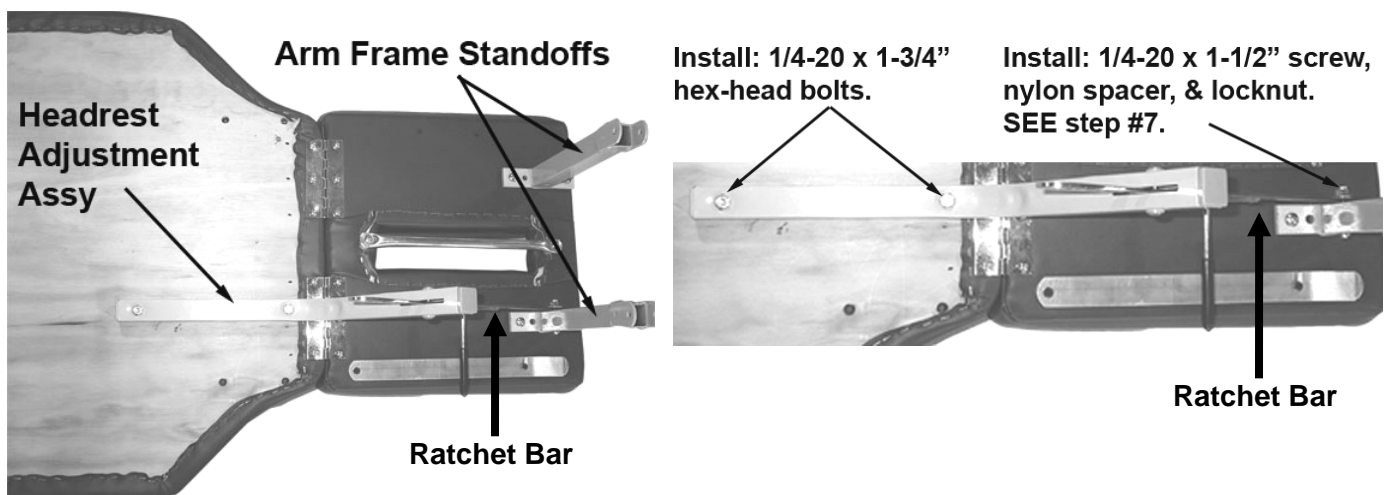

NOTE: Nylon spacer must go between the **Arm Frame Standoff** and the **Ratchet Bar**. Tighten the ¼-20 x 1-½” screw and locknut to snug, then loosen ¼ turn. The **Ratchet Bar** must be able to move for headrest adjustment.

NOTE: When adjusting the headrest, be sure to provide support with your hand, or it will drop rapidly.

Leg Installation

8. Use your 7/16” wrench to remove (8) ¼-20 x 3” Hex-head Bolts from the two (2) **Table Frame Rails**. You will use these same bolts to install the legs in following steps.
 9. Align the (2) **Table Angled Legs** with holes in **Table Frame Rails**. Then, insert the (previously removed) (8) ¼-20 x 3” hex head bolts and leave them slightly loose. **Table Angled Leg** with WINCO decal should go on “headrest end” of table.
 10. Assemble the **Cross-brace** to the **Table Angled Legs** (see next NOTE) using (4) ¼-20 x 1-½” screws and (4) ¼-20 “Nylock” lock nuts. Heads of screws should face table ends for a better appearance. Apply lock nuts and **tighten with a wrench**.
- NOTE:** **Cross-Brace** is only welded on three sides at each end. Position it so that the un-welded side is facing tabletop. (For a better appearance.)
11. **Tighten all of the (8) ¼-20 x 3” hex head bolts with a wrench**, that you previously left slightly loose on the **Table Angled Legs**.
 12. Using at least two people and turn table upright.
 13. If table is not level, loosen or tighten **Glide Feet (leveling feet)** until table is level.

Optional Shelf Installation

14. If your table has a shelf, double-sided tape has been installed on the cross brace of the Welded Leg Assemblies.
15. Before removing the paper on the tape, make sure you “test fit” the shelf.
16. Once you are certain where you want to position the shelf, peel off the paper to expose adhesive.


*After peeling paper from adhesive tape:
Use damp sponge to moisten tape to allow shelf to be adjusted during install.
Once tape is dry shelf will stay in place.*

17. Carefully position shelf in place on adhesive.

Wrinkles in Vinyl – Storage and shipping can cause wrinkles of the vinyl on your Winco table. This is normal. Please allow 1-2 weeks at room temperature (approx. 70 degrees) to allow for vinyl to expand back to original appearance.

Do not over tighten nuts and bolts. Periodically check all fasteners for tightness.

Parts Guide & List

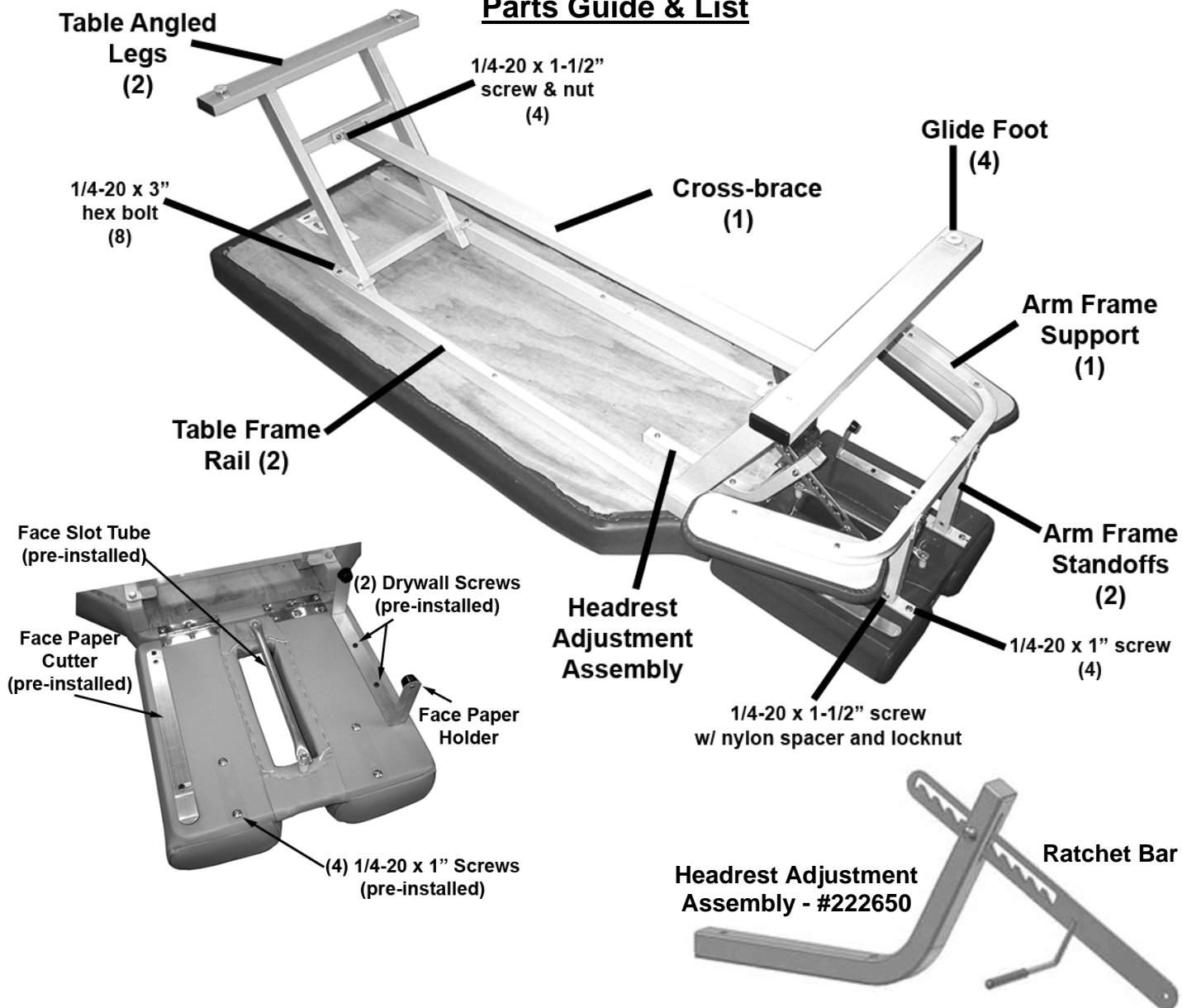

Items factory installed on table:

Qty.

Sizing chart – part number

- Face Slot Tube
- Face Paper Cutter
- Table Frame Rails
- Shelf (optional)
- 4" Hinge
- 1" x 2" rectangular insert caps, (gray)
- Upholstered Top
- Upholstered Face Rest

- 1
- 1
- 2
- 1
- 2
- 4
- 1
- 1

- 219140
- 219160
- 222600
- 267115
- 301800
- 701010
- Call for special part number
- Call for special part number

1" drywall screws (in bottom of Face Rest)
face paper holder to Face Rest


2



566540

1/4-20 x 1" Screw
(see bottom of Face Rest)
arm-frame assembly to Face Rest

4

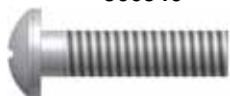

566517

1/4-20 x 3" Hex-head Bolt
front & rear leg assys. to table frame rail

8



566408

Parts List (continued)

Items included with table (NOT factory installed):

Table Legs

Standard Height (22") Legs	2	221210
Special height Legs	2	Call For Part Numbers
Glide Feet (leveling feet)	4	500350
Rectangular insert caps (black, on legs)	4	701008
Glide Foot Insert	4	221235
Cross-brace	1	221300

8050 Headrest Adjustment Assembly

Spacer, nylon (1/16 thick)	1	222650
Gray insert cap	1	566312
	1	700700

Arm Frame Assembly

805-860-870 Arm Frame	1	222605
Gray insert caps	2	700700
KD Standoff assy. welded	2	222630
Upholstered Armrest	2	Call for special part number
Plastic Arm Pad Panel	2	261815
Face Paper Holder	1	219100

¼-2- x 1½" Carriage Bolt

(4) welded brace assy. to front & rear leg assy.

4



566401

¼-20 x 1-½" Screw

(1) 805 Ratchet Bar to arm-frame standoff

1



566506

¼-20 x 1-¾" Hex-head Bolt

face-rest lock tube to table
& (2) arm support to standoffs

4



566404

¼-20 Nylon lock nut

(4) welded-brace assy. to front & rear leg assy.
& (2) arm support to standoffs
& (1) arm support to 805 Ratchet Bar

7



566204

Other Quality Winco Medical Furnishings You May Be Interested In:



Clinical Recliners




Exam Tables



Convalescent (LTC) Recliners



Ring Binder Carts

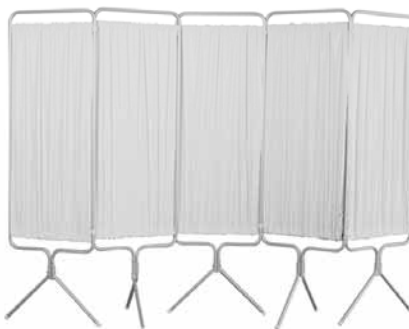

Privacy Screens



Phlebotomy Chairs



Specialty Chair Transfer Chairs



Stools



Room Chairs

Visit us online at www.wincomfg.com



ISO 9001-2008 CERTIFIED



LIMITED WARRANTY

• 5516 SW 1st Lane • Ocala, FL 34474
 • Phone (352) 854-2929 • (800) 237-3377
 • Fax (352) 854-9544
 • www.wincomfg.com
 • customerservice@wincomfg.com

TERMS

Winco Mfg., LLC. warrants this product to be free of manufacturer's defects in material and workmanship, provided that the product is used according to normal operating conditions and proper maintenance intended by manufacturer. This warranty is available only to the original retail purchaser, is non-transferable and commences on the date of retail sale; proof of purchase required.

Limited One Year Complete Product Warranty: Winco Mfg., LLC warrants the complete product for one (1) year. At Winco Mfg., LLC. sole discretion, it may repair or replace any components freight free that are found to be defective during the first year. *Winco Mfg., LLC. shall not be liable for any labor, or any other costs incurred as a result of or in conjunction with a warranty claim.*

Limited Three Year Warranty: Winco Mfg., LLC warrants the recliner mechanisms, electrical components, vinyl [from cracking or delaminating] pressurized gas springs, when new, for a period of three (3) years. At Winco Mfg., LLC. sole discretion, it may repair or replace components found to be defective. *Winco Mfg., LLC. shall not be liable for any labor, shipping or any other costs incurred as a result of or in conjunction with a warranty claim.*

Limited Lifetime Chair Frame Warranty: Winco Mfg., LLC warrants the steel chair frame, when new, for the lifetime of the original purchaser. This limited warranty does not apply to paint/finish or any components attached to the frame such as; upholstery, foam, casters, mechanisms or related parts that are covered under above warranties. *Winco Mfg., LLC shall not be liable for any labor, shipping or any other costs incurred as a result of or in conjunction with a warranty claim.*

The purchaser's exclusive remedy under this warranty shall be limited to such repair or replacement of defective components at Winco Mfg., LLC sole discretion. For warranty service, contact the dealer that the product was originally purchased from or Winco Mfg., LLC directly.

EXCLUSIONS

There are no other warranties, conditions, representations or guarantees, express or implied, made or intended by Winco Mfg., LLC and all other warranties, conditions, representations or guarantees including any warranties, conditions, representations or guarantees under any Sale of Good Act or Like legislation or statute is hereby expressly excluded. Any and all other implied warranties shall not extend beyond the duration of the express warranty. Liability for incidental or consequential damages is excluded to the extent permitted by law. Some states do not allow the exclusion or limitation so the above limitation or exclusion may not apply to you. This warranty gives you specific legal rights. In addition you may also have other rights, which vary from state to state.

GENERAL PROVISIONS

No warranty herein contained or set out shall apply when damage or repair is caused by any of the following:

1. Damage in transit.
2. Accident, alteration, abuse or misuse of product ie. exceeding weight capacities, applying inordinate pressure to footrest/leg rest, use in Mental Health facilities or like institutions.
3. Fire, water damage, theft, war, riot, hostility, acts of God.



Examples contained in this list are not to be construed as all-inclusive.