

8400 Pediatric Table & 8450 Multi-Use Table



CUSTOMER INSTRUCTIONS

PLEASE READ AND FAMILIARIZE YOURSELF WITH ALL INSTRUCTIONS BEFORE USING THIS PRODUCT. If you have trouble understanding these instructions contact your dealer or Winco customer support, (800) 237-3377 before attempting to use this product; otherwise injury may occur.

Winco assumes no responsibility for damage or injury caused by improper assembly, installation, use, or maintenance of these products.



1. READ AND FOLLOW ALL DIRECTIONS.
2. NEVER stand on table or INJURY MAY OCCUR.
3. NEVER allow user to sleep unattended on table or INJURY MAY OCCUR.
4. NEVER leave an unconscious user unattended on table or INJURY MAY OCCUR.
5. DO NOT use table for transporting user in or with ANY type of vehicle or trailer. Winco tables have not been tested or approved for use by an occupant in any type of vehicle or trailer.
6. NEVER leave an infant or child unattended on table or INJURY MAY OCCUR.
7. DO NOT allow children to play on or with the table or INJURY MAY OCCUR.
8. DO NOT put hands, feet or clothing into any openings on table. Attendant MUST confirm that users hands and feet are safe while positioning patient or INJURY MAY RESULT.
9. Periodically, recheck tightness of all nuts, bolts and screws.
10. USE ONLY WINCO AUTHORIZED REPLACEMENT PARTS.
11. NEVER EXCEED the recommended **weight capacity of 400 lbs.** (181.4 kg).
12. SAVE THESE INSTRUCTIONS for future reference and training.

8400-8450 Weight Capacity = 400 lbs. (181.4 kg.)

Tools needed:

- Phillips-head screwdriver
- 7/16" wrench (multiple size wrench included)
- Scissors or box cutter
- - If using powered tools always start hardware together by hand first -

PLEASE READ AND FAMILIARIZE YOURSELF WITH ALL INSTRUCTIONS BEFORE PROCEEDING WITH ASSEMBLY

SEE "Parts Guide & List" (pg.3) for visual reference during assembly.

1. After checking your product for any shipping damage, table needs to be positioned upside-down on cardboard or protective surface. Remove all packing material and hardware that was secured for shipping. Cut tape on corners of box so that cardboard will lay flat while you assemble table.

Leg Installation for 8400-8450:

2. Find the two **Table Legs** that you will be installing to the **Table Frame Rails** and the threaded holes in the wood table bottom.
3. One at a time, position the (2) **Table Legs** between **Table Frame Rails** (as shown in parts guide), and continue to step #4.
4. Install (2) ¼-20 x 1-½" screws through the upper brace on each of the (2) **Table Legs** and into the (2) threaded holes on bottom of **Upholstered Top**.

NOTE: **Do not** fully tighten at this time.

5. Secure the **Table Legs** to the **Table Frame Rails** with (4) ¼-20 x 2-½" screws and ¼-20 nuts.

NOTE: **Do not** fully tighten at this time.

Brace Installation for 8400-8450:

1. Place (4) ¼-20 x 1-½” carriage bolts in (4) holes through the lower brace of the Table Legs.
2. Align **the Cross-brace** (smooth, un-welded side, facing table bottom) with the (4) screws & holes on both **Table Legs** and fasten with (4) ¼-20 nuts. Tighten these (4) screws.

NOTE: For table without shelf; nylon inserts on brace should face toward bottom of table.

3. **8450 ONLY:** TIGHTEN ALL nuts & bolts at this time.
4. Using at least two people, turn table upright.
5. **8450 ONLY:** Install **Paper Holder** and paper by inserting it through the **Paper Rod Holders** on the ends of **Table Frame Rails**.
6. Adjust the (4) **Glide Feet** to level table.

**WARNING**

***Do not over tighten nuts and bolts.
Periodically recheck all fasteners for tightness.***

Optional Shelf installation for 8400-8450:

1. Double-sided tape has been installed on the **Cross-brace**.
2. Before removing paper on tape, “test fit” the shelf. Shelf should be centered on Welded Brace Assembly.
3. Once you are certain of where you will position the shelf, peel off the paper to expose adhesive.

*After peeling paper from adhesive tape:
Use damp sponge to moisten tape to allow shelf to be adjusted during install.
Once tape is dry shelf will stay in place.*

4. Carefully position shelf in place on adhesive.

ADDITIONAL INSTRUCTIONS for 840 PEDIATRIC TABLE ONLY:

Rear Pediatric Retainer Installation:

1. With table laying on one long side (be sure to protect vinyl); remove the (4) four ¼-20 x 3" screws & (4) ¼ - 1" washers from the **Table Frame Rail**.

NOTE: The 3" screws are left loose intentionally, so you can identify them easier.

2. Place the screws & washers (you just removed) through the brackets on the **Rear Pediatric Retainer**, then through the **Table Frame Rail**, and finally back into the threaded holes in the table top.

The washers should be positioned between the bracket and the **Table Frame Rail**.

TIP: Start one screw and allow it to hold the retainer while you start the other screws.)

3. Once all (4) screws have been installed, tighten them.

Side Pediatric Retainers Installation:

1. Next, install (3) ¼-20 x 1-¾" bolts (hex-head) through the (3) brackets on the **Side Pediatric Retainers**, then through (3) **Nylon Spacers**, and finally into the threaded holes in the table top.

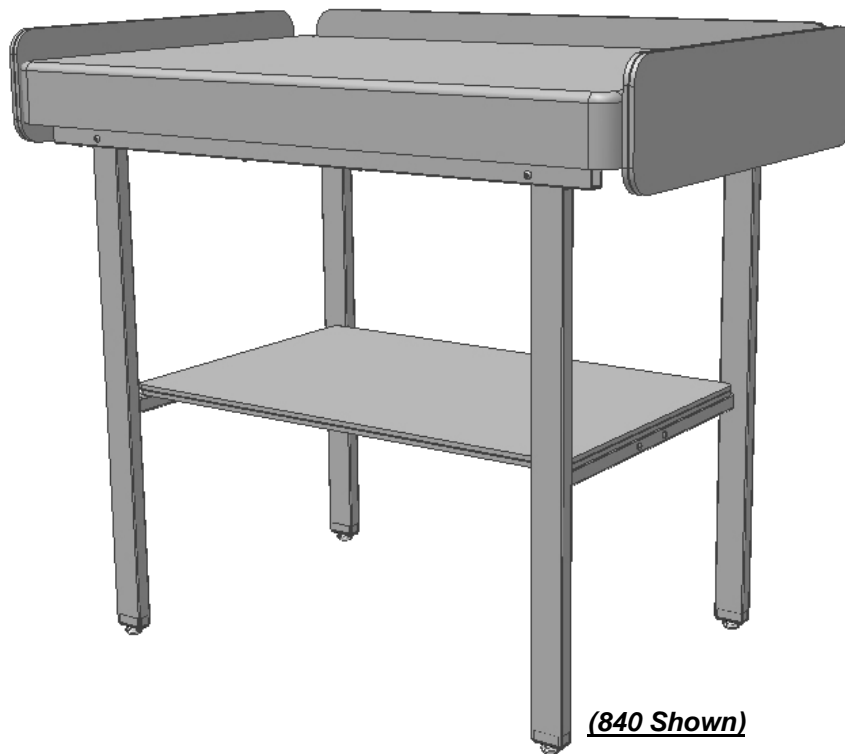
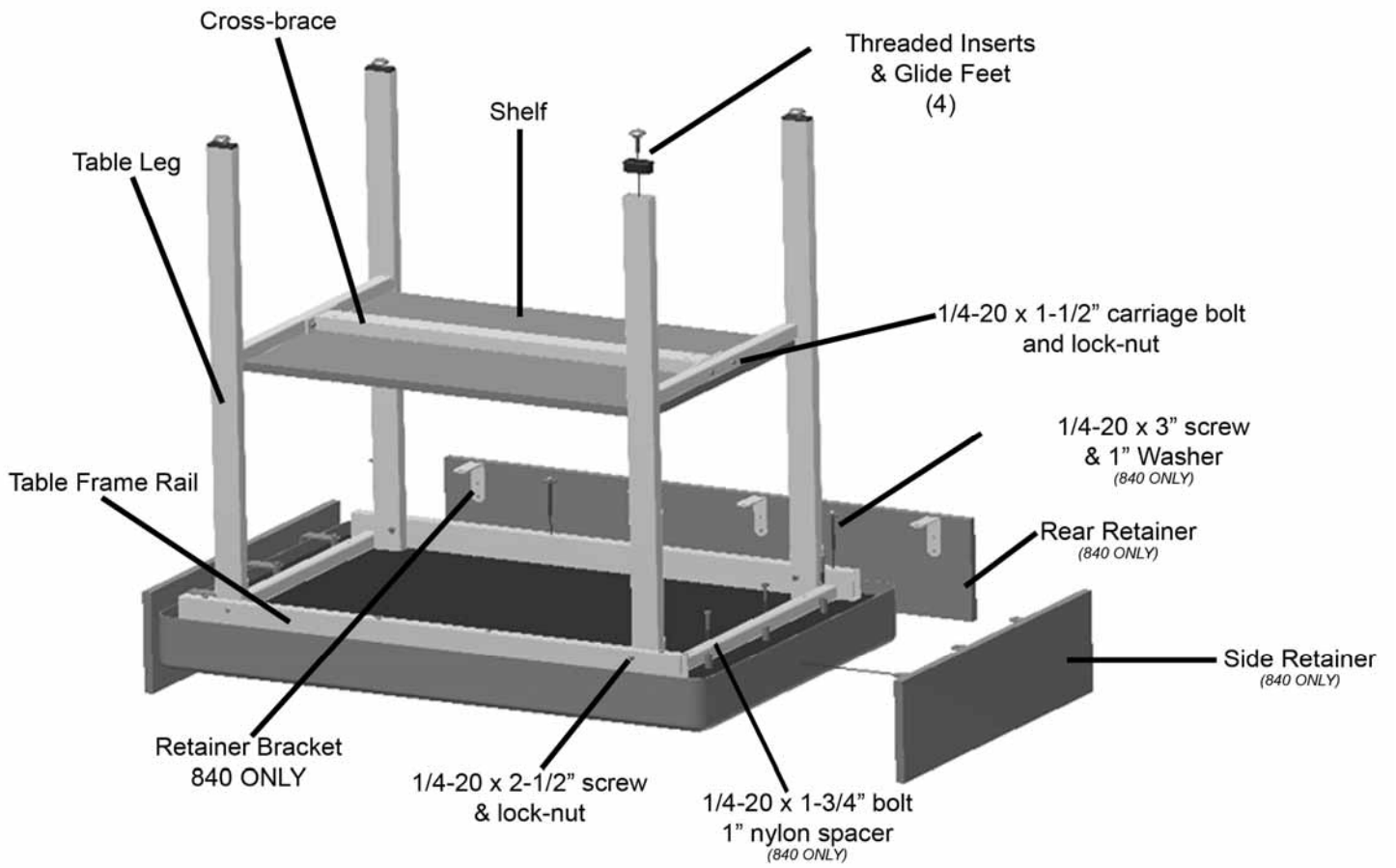
TIP: Thread all bolts in by hand before fully tightening them.

2. TIGHTEN ALL nuts & bolts at this time.





Vinyl Strap Adjustment – 8450 Only - Should the vinyl strap on your Winco table need to be adjusted, unscrew the screw that currently holds either side of it in place. Pierce another hole in the strap to give desired tension and re-insert screw.

Wrinkles in Vinyl – Storage and shipping can cause wrinkles of the vinyl on your Winco table. This is normal. Please allow 1-2 weeks at room temperature (approx. 70+ degrees) to allow for vinyl to expand back to original appearance.

Parts Guide



Parts List

| <u>Items factory installed on table:</u> | <u>Qty.</u> | <u>Sizing chart – part number</u> |
|--|-------------|--|
| Table Frame Rails | 2 | 228401 |
| Model 840 = ¼-20 x 3" screw | 4 | 566507 |
| ¼ x 1" washer | 4 | 566306 |
| ¼-20 x 2-½" screw | 4 | 566519 |
| Model 845 = ¼-20 x 2-½" screw | 8 | 566519 |
| 1" x 2" rectangular insert caps, (gray) | 4 | 701010 |
| Upholstered Top | 1 | CALL FOR P/N |
| Model 8450, Vinyl Strap (for paper) | 1 | 271500 |
| Model 8450, Paper Rod Clamp (bracket) | 2 | 501810 |
| <u>Items included with table (not factory installed):</u> | | |
| Table Legs | | |
| Model 8400 Standard height 36" | 2 | 222711 |
| Model 8450 Standard height 30" | 2 | 222710 |
| Glide Feet (leveling feet) | 4 | 500350 |
| Threaded Insert Caps (black) | 4 | 500105 |
| Cross-brace w/ nylon inserts | 1 | 228403 |
| Model 8450, Paper Holder Rod | 1 | 222705 |
| ½" plastic cap, white | 2 | 700650 |
| ¼-20 lock nut | 8 | 566204 |
| welded brace assy. to both leg assys. & leg assys. to table frame rails | |  |
| ¼ - 20 x 1½ carriage bolt | 4 | 566401 |
| leg assy. to cross brace | |  |
| ¼-20 x 1-½ screw | 4 | 566506 |
| leg assy. to table bottom | |  |
| ¼-20 x 2-½ screw | 4 | 566519 |
| leg assys. to table frame rails w/ ¼-20 lock nuts | |  |
| <hr/> | | |
| Decals (set) | 1 | 007500 |
| Pediatric Retainer Bracket (8400 ONLY) | 10 | 228407 |
| Rear Pediatric Retainer (8400 ONLY) | 1 | 268403 |
| Bolts & Washers (see Table Frame 8400 above) | | |
| Side Pediatric Retainer (8400 ONLY) | 2 | 268415 |
| ¼-20 x 1-¾" bolt | 6 | 566404 |
| 1" Nylon spacers | 6 | 566315 |
| Mylar Tape Measure ¾" x 40" | 1 | 705840 |

Other Quality Winco Medical Furnishings You May Be Interested In:



Clinical Recliners



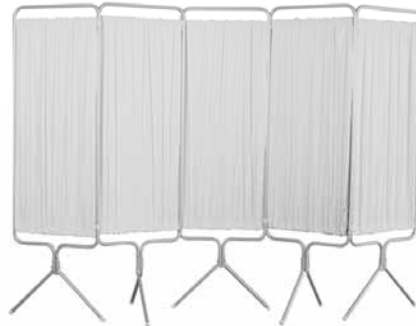
Exam Tables



Convalescent (LTC) Recliners



Ring Binder Carts



Privacy Screens



Phlebotomy Chairs



Specialty Chair Transfer Chairs



Stools



Room Chairs

Visit us online at www.wincomfg.com



ISO 9001-2008 CERTIFIED



LIMITED WARRANTY

• 5516 SW 1st Lane • Ocala, FL 34474
 • Phone (352) 854-2929 • (800) 237-3377
 • Fax (352) 854-9544
 • www.wincomfg.com
 • customerservice@wincomfg.com

TERMS

Winco Mfg., LLC. warrants this product to be free of manufacturer's defects in material and workmanship, provided that the product is used according to normal operating conditions and proper maintenance intended by manufacturer. This warranty is available only to the original retail purchaser, is non-transferable and commences on the date of retail sale; proof of purchase required.

Limited One Year Complete Product Warranty: Winco Mfg., LLC warrants the complete product for one (1) year. At Winco Mfg., LLC. sole discretion, it may repair or replace any components freight free that are found to be defective during the first year. *Winco Mfg., LLC. shall not be liable for any labor, or any other costs incurred as a result of or in conjunction with a warranty claim.*

Limited Three Year Warranty: Winco Mfg., LLC warrants the recliner mechanisms, electrical components, vinyl [from cracking or delaminating] pressurized gas springs, when new, for a period of three (3) years. At Winco Mfg., LLC. sole discretion, it may repair or replace components found to be defective. *Winco Mfg., LLC. shall not be liable for any labor, shipping or any other costs incurred as a result of or in conjunction with a warranty claim.*

Limited Lifetime Chair Frame Warranty: Winco Mfg., LLC warrants the steel chair frame, when new, for the lifetime of the original purchaser. This limited warranty does not apply to paint/finish or any components attached to the frame such as; upholstery, foam, casters, mechanisms or related parts that are covered under above warranties. *Winco Mfg., LLC shall not be liable for any labor, shipping or any other costs incurred as a result of or in conjunction with a warranty claim.*

The purchaser's exclusive remedy under this warranty shall be limited to such repair or replacement of defective components at Winco Mfg., LLC sole discretion. For warranty service, contact the dealer that the product was originally purchased from or Winco Mfg., LLC directly.

EXCLUSIONS

There are no other warranties, conditions, representations or guarantees, express or implied, made or intended by Winco Mfg., LLC and all other warranties, conditions, representations or guarantees including any warranties, conditions, representations or guarantees under any Sale of Good Act or Like legislation or statute is hereby expressly excluded. Any and all other implied warranties shall not extend beyond the duration of the express warranty. Liability for incidental or consequential damages is excluded to the extent permitted by law. Some states do not allow the exclusion or limitation so the above limitation or exclusion may not apply to you. This warranty gives you specific legal rights. In addition you may also have other rights, which vary from state to state.

GENERAL PROVISIONS

No warranty herein contained or set out shall apply when damage or repair is caused by any of the following:

1. Damage in transit.
2. Accident, alteration, abuse or misuse of product ie. exceeding weight capacities, applying inordinate pressure to footrest/leg rest, use in Mental Health facilities or like institutions.
3. Fire, water damage, theft, war, riot, hostility, acts of God.

Examples contained in this list are not to be construed as all-inclusive.

