

8400 Pediatric Table
8450 Multi-Use Table
with Cabinet or Drawer Modules



CUSTOMER INSTRUCTIONS

PLEASE READ AND FAMILIARIZE YOURSELF WITH ALL INSTRUCTIONS BEFORE USING THIS PRODUCT. If you have trouble understanding these instructions contact your dealer or Winco customer support, (800) 237-3377 before attempting to use this product; otherwise injury may occur.

Winco assumes no responsibility for damage or injury caused by improper assembly, installation, use, or maintenance of these products.



1. READ AND FOLLOW ALL DIRECTIONS.
2. NEVER stand on table or INJURY MAY OCCUR.
3. NEVER allow user to sleep unattended on table or INJURY MAY OCCUR.
4. NEVER leave an unconscious user unattended on table or INJURY MAY OCCUR.
5. DO NOT use table for transporting user in or with ANY type of vehicle or trailer. Winco tables have not been tested or approved for use by an occupant in any type of vehicle or trailer.
6. NEVER leave an infant or child unattended on table or INJURY MAY OCCUR.
7. DO NOT put hands, feet or clothing into any openings when opening/closing drawers on table. Attendant MUST confirm that users hands and feet are clear of any moving parts before and while opening drawers or INJURY MAY RESULT.
8. Periodically, recheck tightness of all nuts, bolts and screws.
9. USE ONLY WINCO AUTHORIZED REPLACEMENT PARTS.
10. NEVER EXCEED the recommended **weight capacity of 400 lbs.** (181.4 kg).
11. SAVE THESE INSTRUCTIONS for future reference and training.

8400-8450 Weight Capacity = 400 lbs. (181.4 kg.)

Tools needed:

- Phillips Head Screwdriver
- 7/16" wrench, adjustable wrench, or 7/16" socket & ratchet wrench
- Cardboard shipping container (stays under table during assembly to protect upholstery)
- Scissors or box cutter
- - If using powered tools always start hardware together by hand first & wear safety goggles

PLEASE READ AND FAMILIARIZE YOURSELF WITH ALL INSTRUCTIONS BEFORE PROCEEDING WITH ASSEMBLY

1. After checking your product for any shipping damage, remove all packing material and hardware that was secured for shipping.
2. Using at least two (2) people, lift the table off of the skid.
3. Install the included **Glide Feet** into the **Threaded Inserts**, located in bottom of table legs. (see note)

NOTE: Turning table on side or vinyl side down will ease this installation, however, drawers should be removed. Please protect your **Upholstered Top** with cardboard, carpeted floor, etc... when performing this operation. Using two or more people is highly recommended!

4. With table in the upright position, adjust the (4) **Glide Feet** to level table.
5. Check all nuts and bolts to ensure they are tight and did not loosen during shipping.


Model 8450 only:

6. Load paper roll onto **Paper Holder Rod** by removing one of the white plastic caps on the **Paper Holder Rod**. Pull rod out of one **Paper Rod Clamp** and insert through paper roll. Feed rod back into **Paper Rod Clamp** and replace white plastic cap.

Vinyl Strap Adjustment – Should the vinyl strap on your Winco table need to be adjusted, unscrew the screw that currently holds either side of it in place. Pierce another hole in the strap to give desired tension and re-insert screw.

**WARNING**

Do not over tighten nuts and bolts. Periodically recheck all fasteners for tightness.

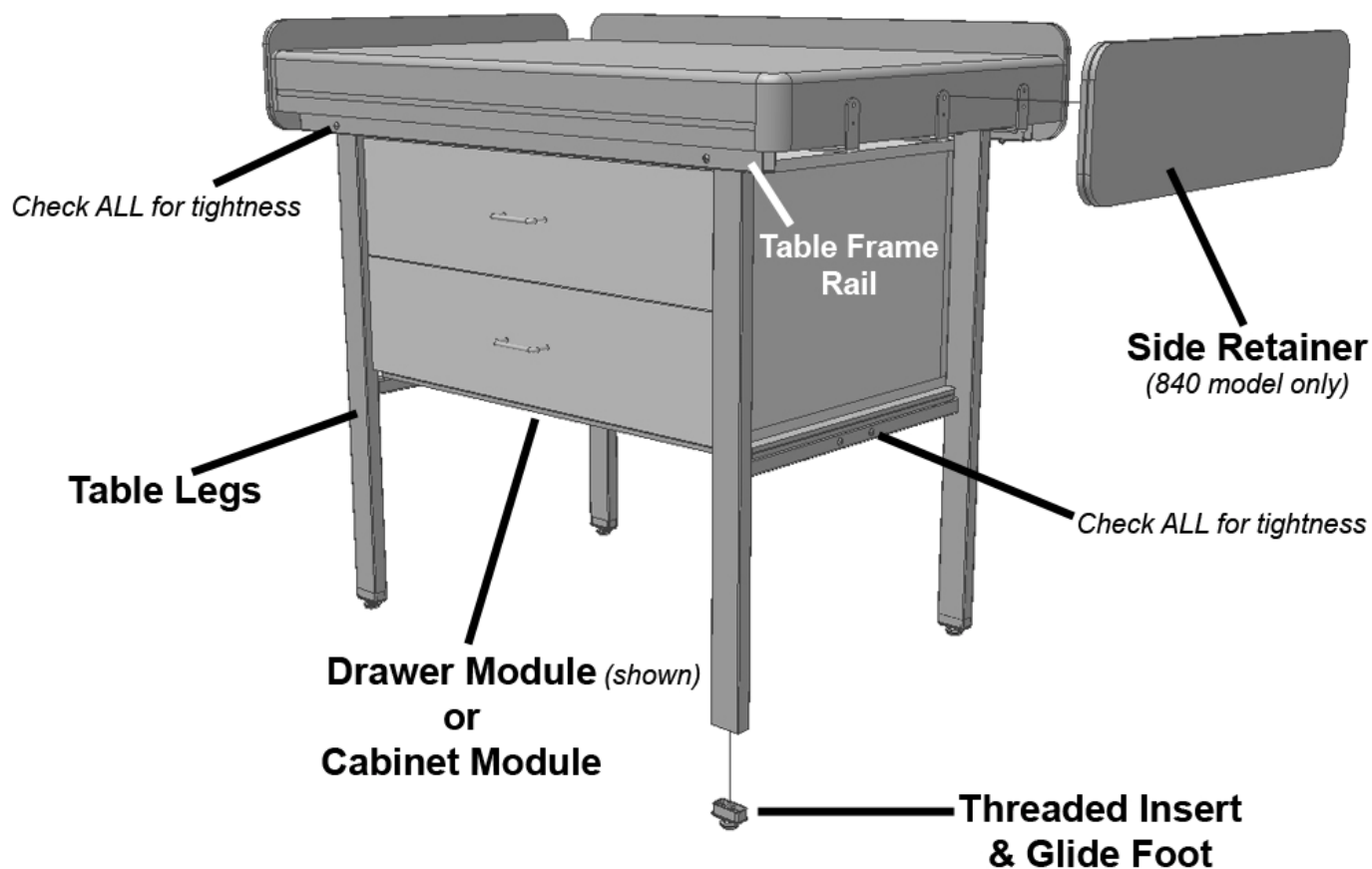

<u>Items factory installed:</u>	<u>Qty.</u>	<u>Part Number</u>
Table Frame Rail	2	228401
Model 840 = 1/4-20 x 3" screw	4	566507
1/4 x 1" washer	4	566306
1/4-20 x 2-1/2" screw	4	566519
Model 845 = 1/4-20 x 2-1/2" screw	8	566519
Upholstered Top	1	CALL FOR P/N
Cross-brace	1	228403
Table Legs		
Model 840 Standard height 36"	2	222711
Model 845 Standard height 30"	2	222710
Threaded Insert Caps (black)	4	500105
<u>Model 840 ONLY</u>		
Pediatric Retainer Bracket (8400 ONLY)	10	228407
Rear Pediatric Retainer (8400 ONLY)	1	268403
Side Pediatric Retainer (8400 ONLY)	2	268415
1" Nylon spacers	6	566315
1/4-20 x 1-3/4" hex-head bolt	6	566404
<u>Model 845 ONLY</u>		
Vinyl Strap (for paper on 8450 only)	1	271500
Paper Holder Rod (8450 only)	1	222705
1/2" plastic cap, white (8450 only)	2	700650

Items NOT factory installed:

Glide Feet (leveling feet)	4	500350
<u>Model 840 ONLY</u>		
Decals (set)	1	007500
Mylar Tape Measure 3/4" x 40"	1	705840





LIMITED WARRANTY

• 5516 SW 1st Lane • Ocala, FL 34474
 • Phone (352) 854-2929 • (800) 237-3377
 • Fax (352) 854-9544
 • www.wincomfg.com
 • customerservice@wincomfg.com

TERMS

Winco Mfg., LLC. warrants this product to be free of manufacturer's defects in material and workmanship, provided that the product is used according to normal operating conditions and proper maintenance intended by manufacturer. This warranty is available only to the original retail purchaser, is non-transferable and commences on the date of retail sale; proof of purchase required.

Limited One Year Complete Product Warranty: Winco Mfg., LLC warrants the complete product for one (1) year. At Winco Mfg., LLC. sole discretion, it may repair or replace any components freight free that are found to be defective during the first year. *Winco Mfg., LLC. shall not be liable for any labor, or any other costs incurred as a result of or in conjunction with a warranty claim.*

Limited Three Year Warranty: Winco Mfg., LLC warrants the recliner mechanisms, electrical components, vinyl [from cracking or delaminating] pressurized gas springs, when new, for a period of three (3) years. At Winco Mfg., LLC. sole discretion, it may repair or replace components found to be defective. *Winco Mfg., LLC. shall not be liable for any labor, shipping or any other costs incurred as a result of or in conjunction with a warranty claim.*

Limited Lifetime Chair Frame Warranty: Winco Mfg., LLC warrants the steel chair frame, when new, for the lifetime of the original purchaser. This limited warranty does not apply to paint/finish or any components attached to the frame such as; upholstery, foam, casters, mechanisms or related parts that are covered under above warranties. *Winco Mfg., LLC shall not be liable for any labor, shipping or any other costs incurred as a result of or in conjunction with a warranty claim.*

The purchaser's exclusive remedy under this warranty shall be limited to such repair or replacement of defective components at Winco Mfg., LLC sole discretion. For warranty service, contact the dealer that the product was originally purchased from or Winco Mfg., LLC directly.

EXCLUSIONS

There are no other warranties, conditions, representations or guarantees, express or implied, made or intended by Winco Mfg., LLC and all other warranties, conditions, representations or guarantees including any warranties, conditions, representations or guarantees under any Sale of Good Act or Like legislation or statute is hereby expressly excluded. Any and all other implied warranties shall not extend beyond the duration of the express warranty. Liability for incidental or consequential damages is excluded to the extent permitted by law. Some states do not allow the exclusion or limitation so the above limitation or exclusion may not apply to you. This warranty gives you specific legal rights. In addition you may also have other rights, which vary from state to state.

GENERAL PROVISIONS

No warranty herein contained or set out shall apply when damage or repair is caused by any of the following:

1. Damage in transit.
2. Accident, alteration, abuse or misuse of product ie. exceeding weight capacities, applying inordinate pressure to footrest/leg rest, use in Mental Health facilities or like institutions.
3. Fire, water damage, theft, war, riot, hostility, acts of God.

Examples contained in this list are not to be construed as all-inclusive.

