

8900-8910

Bariatric Treatment Table



CUSTOMER INSTRUCTIONS

PLEASE READ AND FAMILIARIZE YOURSELF WITH ALL INSTRUCTIONS BEFORE USING THIS PRODUCT. If you have trouble understanding these instructions contact your dealer or Winco customer support, (800) 237-3377 before attempting to use this product; otherwise injury may occur.

Winco assumes no responsibility for damage or injury caused by improper assembly, installation, use, or maintenance of these products.



1. READ AND FOLLOW ALL DIRECTIONS.
2. NEVER stand on table or INJURY MAY OCCUR.
3. NEVER allow user to sleep unattended on table or INJURY MAY OCCUR.
4. NEVER leave an unconscious user unattended on table or INJURY MAY OCCUR.
5. DO NOT use table for transporting user in or with ANY type of vehicle or trailer. Winco tables have not been tested or approved for use by an occupant in any type of vehicle or trailer.
6. NEVER leave an infant or child unattended on table or INJURY MAY OCCUR.
7. DO NOT put hands, feet or clothing into any openings when changing user positions on table. Attendant MUST confirm that users hands and feet are safe while changing positions or INJURY MAY RESULT.
8. Periodically, recheck tightness of all nuts, bolts and screws.
9. USE ONLY WINCO AUTHORIZED REPLACEMENT PARTS.
10. NEVER EXCEED the recommended **weight capacity of 1000 lbs.** (453.6 kg).
11. SAVE THESE INSTRUCTIONS for future reference and training.

8900-8910 Weight Capacity = 1000 lbs. (453.6 kg.)

Tools needed:

- Phillips-head screwdriver
- 7/16" wrench, (multiple size wrench included)
- Scissors or box cutter
- Cardboard shipping container (stays under table during assembly to protect upholstery)
- - If using powered tools always start hardware together by hand first & wear safety goggles

PLEASE READ AND FAMILIARIZE YOURSELF WITH ALL INSTRUCTIONS BEFORE PROCEEDING WITH ASSEMBLY

1. After checking your product for any shipping damage, table needs to be positioned upside-down on cardboard or protective surface. Remove all packing material and hardware that was secured for shipping. Cut tape on corners of box so that cardboard will lay flat while you assemble table.

Leg Assemblies (3):

2. You will find (3) **Welded Leg Assemblies**, in-between or attached-to the **Table Frame Rails**. Pivot up the (2) **Welded Leg Assemblies**, on ends of table (see photo). Insert (2) ¼-20 x 2" screws through (2) holes in each Upper Brace and into (2) threaded holes in the wood of the **Upholstered Top**

NOTE: *Upper braces on all **Welded Leg Assemblies** have (2) holes that need to align with two threaded holes in the wood of the **Upholstered Top**.*

The center **Welded Leg Assembly** must have two sets of holes aligned. Holes in the **Upper Brace of Welded Leg Assembly** must align with holes in the bottom of table, **AND** the oval-shaped holes on the legs must align with holes on both frame rails on the table.

The center **Welded Leg Assy.** may need to be rotated 180 degrees so that the above alignment is possible.

- Secure the middle **Welded Leg Assembly** to the **Table Frame Rails** with (2) $\frac{1}{4}$ -20 x 2-1/2" screws & locknuts. Then, insert (2) $\frac{1}{4}$ -20 x 2" screws through (2) holes in each **Upper Brace** and into (2) threaded holes in the wood of the **Upholstered Top**, see photo.

NOTE: Do not fully tighten at this time.

Welded Braces (2):

Before installing: Make sure that the smooth surface of the **Reinforced Welded Brace Assemblies** will face upward (when the table is in the upright position) prior to fastening to **Lower Leg Brace of Welded Leg Assemblies**. (IMPORTANT for best appearance and proper shelf installation)

- End Legs:** Place (4) $\frac{1}{4}$ -20 x 1-1/2" carriage bolts into the holes in the **Lower Leg Braces** and through the holes on the **Welded Brace Assemblies**. Install locknuts (hand-tighten). See photo. Heads of screws should face the ends of table, for better appearance.
- Center Leg:** Place (2) $\frac{1}{4}$ -20 x 2" screws into the holes in the **Lower Leg Brace** and through the holes in **BOTH** of the **Welded Brace Assemblies**. Install locknuts. See photo.
- TIGHTEN ALL FASTENERS AT THIS TIME** including the nuts & screws holding the **Welded Leg Assemblies** to the **Table Frame Rails**.
- Install the six (6) **Glide Feet** into the threaded ends of the table legs.

Additional instructions for 8910 models ----- (8900 model go to step 11)

- Attach the **Face Slot Tube** so that one hole is aligned with the center hole of the **Upper Brace**, (see photo, flat side of Face Slot Tube faces table bottom). Position the opposite end of the **Face Slot Tube** over the 1" nylon spacer and the hole in table bottom. Install two (2) 1-3/4" Panhead Phillips screws through both ends of the **Face Slot Tube**, then tighten into the wood table bottom with a Phillips Head Screwdriver.
- Install **Face Paper Holder** using (2) 1" "drywall-type" screws in pre-drilled holes on either side of Face Slot.
- Install **Face Paper Cutter** on **Table Frame Rail** (where desired) opposite **Face Paper Holder**. Peel backing from tape on back of cutter, press cutter firmly into place.

*HINT: After peeling paper from adhesive tape:
Use damp sponge to moisten tape to allow face paper cutter to be adjusted during install.
Once tape is dry face paper cutter will stay in place.*

- Using at least two people, turn table upright.
- Place **Paper Holder Rod** through the (2) **Paper Rod Clamps**, (with paper roll in-between).
- Adjust (4) 5/16" **Adjustable Glides** to level table.

Optional Shelf installation:

- Double-sided tape has been installed on the top of the **Welded Brace Assemblies**.
- Before removing paper on tape, "test fit" the shelf. Shelf should be centered between outside **Welded Leg Assemblies**.
- Once you are certain of where you will position the shelf, peel off the paper to expose adhesive.


*HINT: After peeling paper from adhesive tape:
Use damp sponge to moisten tape to allow shelf to be adjusted during install.
Once tape is dry shelf will stay in place.*

17. Carefully position shelf in place on adhesive.

Wrinkles in Vinyl – Storage and shipping can cause wrinkles of the vinyl on your Winco table. This is normal. Please allow 1-2 weeks at room temperature (approx. 70 degrees) to allow for vinyl to expand back to original appearance.

Do not over tighten nuts and bolts. Periodically check all fasteners for tightness.

890 & 891
*** = 891 Only**

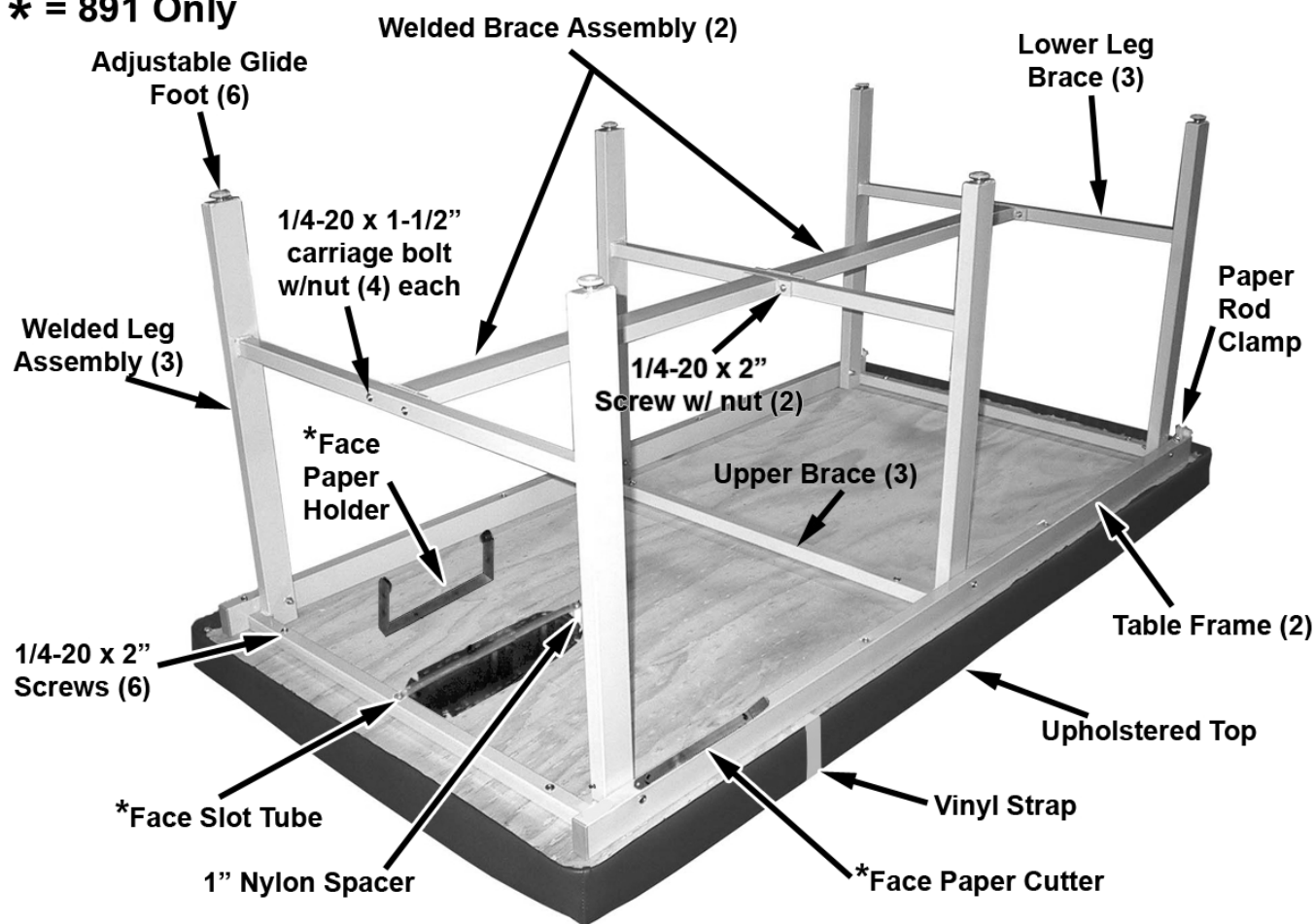

<u>Items factory installed on table:</u>	<u>Qty.</u>	<u>Sizing chart – part number</u>
Table Frame	2	222765
1/4-20 x 3" screw (on paper rod straps)	2	566507
1/4-20 x 2 1/2" screw	4	566519
1/4-20 lock nut	4	566204
Upholstered Top	1	CALL FOR PART NUMBER
1" x 2" rectangular insert caps, (gray)	4	701010

Items included with table (not factory installed):

Welded Leg Assembly (standard 30") (special height leg assembly)	3	222710 CALL FOR PART NUMBER
Glide Feet (leveling feet)	6	500350
Welded Brace Assembly	2	222760

1/4-20 lock nut 8
6 for welded brace assy. to leg assys.
& 2 for welded leg assys to table frame rails



566204

1/4 - 20 x 1 1/2 carriage bolt 4
welded brace assy. to front & rear leg assy.



566401

1/4-20 X 2" screw 8
6 for upper brace to table bottom &
2 for center leg (lower brace) to welded
brace assemblies



566549

1/4-20 X 2-1/2" screw 2
leg assy. to table frame rail



566519

The following additional items for 8910 tables ONLY:

1" long - nylon spacer 1
face slot tube to table bottom



566315

#10 x 1-3/4" Panhead Phillips screw 2
face slot tube to table bottom



566526

Face Slot Tube 1
Face Paper Cutter 1
Face Paper Holder 1

219120

219160

219100

#6 x 1" "drywall-type" screw 2
face paper holder to table bottom



566540



LIMITED WARRANTY

5516 SW 1st Lane • Ocala, FL 34474
 Phone (352) 854-2929 • (800) 237-3377 • Fax (352)
 854-9544

TERMS

Winco Mfg., LLC. warrants this product to be free of manufacturer's defects in material and workmanship, provided that the product is used according to normal operating conditions and proper maintenance intended by manufacturer. This warranty is available only to the original retail purchaser, is non-transferable and commences on the date of retail sale; proof of purchase required.

Limited One Year Complete Product Warranty: Winco Mfg., LLC warrants the complete product for one (1) year. At Winco Mfg., LLC. sole discretion, it may repair or replace any components freight free that are found to be defective during the first year. *Winco Mfg., LLC. shall not be liable for any labor, or any other costs incurred as a result of or in conjunction with a warranty claim.*

Limited Three Year Warranty: Winco Mfg., LLC warrants the recliner mechanisms, electrical components, vinyl [from cracking or delaminating] pressurized gas springs, when new, for a period of three (3) years. At Winco Mfg., LLC. sole discretion, it may repair or replace components found to be defective. *Winco Mfg., LLC. shall not be liable for any labor, shipping or any other costs incurred as a result of or in conjunction with a warranty claim.*

Limited Lifetime Chair Frame Warranty: Winco Mfg., LLC warrants the steel chair frame, when new, for the lifetime of the original purchaser. This limited warranty does not apply to paint/finish or any components attached to the frame such as; upholstery, foam, casters, mechanisms or related parts that are covered under above warranties. *Winco Mfg., LLC shall not be liable for any labor, shipping or any other costs incurred as a result of or in conjunction with a warranty claim.*

The purchaser's exclusive remedy under this warranty shall be limited to such repair or replacement of defective components at Winco Mfg., LLC sole discretion. For warranty service, contact the dealer that the product was originally purchased from or Winco Mfg., LLC directly.

EXCLUSIONS

There are no other warranties, conditions, representations or guarantees, express or implied, made or intended by Winco Mfg., LLC and all other warranties, conditions, representations or guarantees including any warranties, conditions, representations or guarantees under any Sale of Good Act or Like legislation or statute is hereby expressly excluded. Any and all other implied warranties shall not extend beyond the duration of the express warranty. Liability for incidental or consequential damages is excluded to the extent permitted by law. Some states do not allow the exclusion or limitation so the above limitation or exclusion may not apply to you. This warranty gives you specific legal rights. In addition you may also have other rights, which vary from state to state.

GENERAL PROVISIONS

No warranty herein contained or set out shall apply when damage or repair is caused by any of the following:

1. Damage in transit.
2. Accident, alteration, abuse or misuse of product ie. exceeding weight capacities, applying inordinate pressure to footrest/leg rest, use in Mental Health facilities or like institutions.
3. Fire, water damage, theft, war, riot, hostility, acts of God.



Examples contained in this list are not to be construed as all-inclusive.