

890 Bariatric Table Parts List





PARTS LIST

Image Number	Note	Part Number	Description	QTY per Product
1	A	U3009XX	890 UPHOLSTERED TOP	1
1A	B	222765	BARI-TABLE FRAME RAIL 68 IN.	2
1A	B	701010	1X2X18 RECT INSERT CAP/GLIDE LT.GRAY	4
1A	B	566507	SCREW-MACH 1/4-20 X 3 RND PHILLIP M/S ZINC	2
1A	B	566519	SCREW-MACH 1/4-20 X 2 1/2 PAN PHILLIP M/S ZINC	2
1A	B	566528	SCREW-TAP 10 X 2 3/4 PAN PHIL T/S ZINC	4
2		222760	WELDED BRACE ASSY. BARI-TABLE	2
2	C	566506	SCREW-MACH 1/4-20 X 1 1/2 RND PHIL M/S ZINC	4
2	D	566549	SCREW MACH 1/4-20 X 2 RND PHIL M/S ZINC	2
2		566204	NUT-1/4-20 HEX NYLON LOCKNUT ZINC	6
3		263965K	BARIATRIC TABLE SHELF KIT	1
3	B	263965	BARIATRIC TABLE SHELF	1
N/A	B	705600	DOUBLE SIDED FOAM TAPE, 3/4" X 36YDS	1 ft.

NOTES

Note	Description
A	XX- Use color code for specific part number i.e. 02 - Royal Blue
B	These parts are included with the above listed UPHOLSTERED TOPS
C	These screws attach the "welded brace assembly." to the OUTSIDE legs
D	These screws attach the "welded brace assembly." to the MIDDLE leg

890 Bariatric Table Parts List

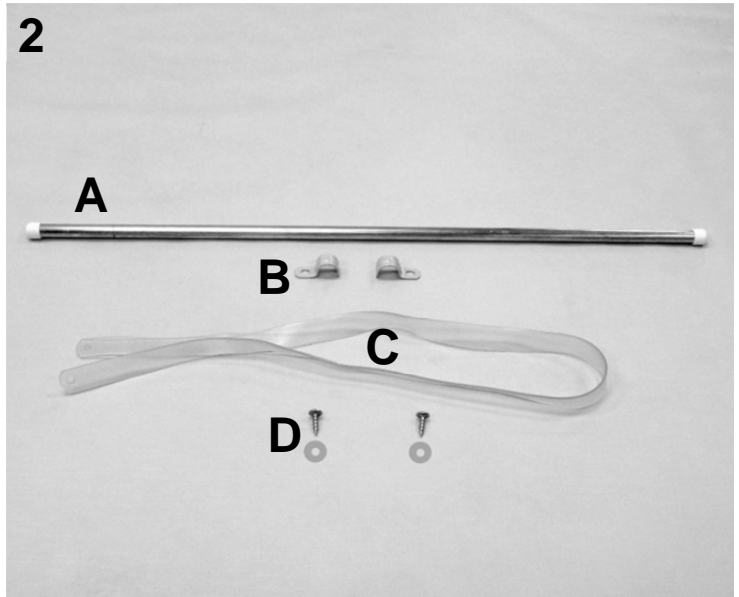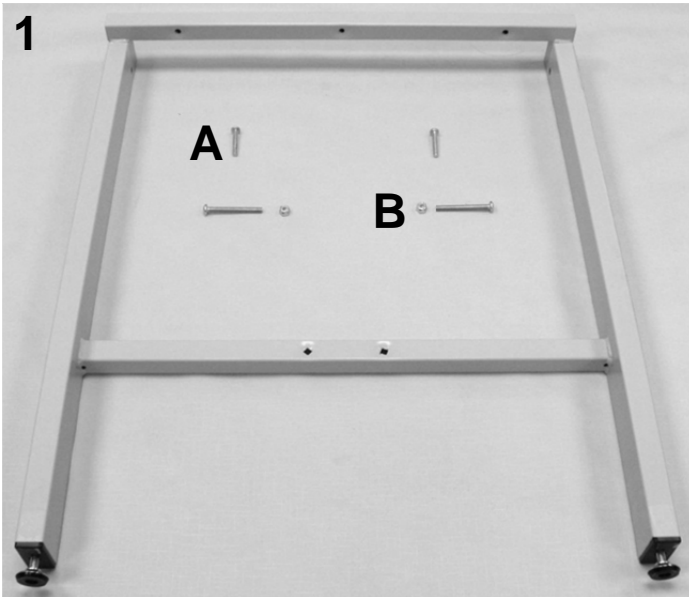

PARTS LIST

Image Number	Note	Part Number	Description	QTY per Product
1		222707K	18" BARI-WELDED TABLE LEG KIT	3
1	A	222707	18" BARI-WELDED TABLE LEG	1
1		222749K	25" BARI - WELDED TABLE LEG KIT	3
1	A	222749	25" BARI - WELDED TABLE LEG	1
1		222750K	30" BARI-WELDED TABLE LEG KIT	3
1	A	222750	30" BARI-WELDED TABLE LEG	1
1	A	500350	GLIDE NON-SWIVEL ZINC 5/16X1" NP LN	2
1	A	500105	INSERT, BLK HDPE, RIBBED 16-18 ga 1X2 , 5/16-18 THREAD	2
1A	A	566549	SCREW MACH 1/4-20 X 2 RND PHIL M/S ZINC	2
1B	A	566519	SCREW-MACH 1/4-20 X 2 1/2 PAN PHILLIP M/S ZINC	2
1B	A	566204	NUT-1/4-20 HEX NYLON LOCKNUT ZINC	2
2A		222770K	BARIATRIC PAPER HOLDER ROD KIT	1
2A	A	222770	BARIATRIC PAPER HOLDER ROD	1
2A	A	700650	1/2" PLASTIC CAP, WHITE	2
2B		201810	PAPER ROD CLAMP, GRAY	2
2C		271505	BARIATRIC, VINYL TABLE STRAP	1
2D		566302	WASHER-#10 SAE F/W ZINC	2
2D		566534	SCREW-TAP 10 X 3/4 RND WASHER SQ T/S ZINC	2

NOTES

Note	Description
A	These parts are included in ALL above listed kit.