


PARTS LIST

| Image Number | Note | Part Number | Description | QTY per Product |
|--------------|------|----------------|--|-----------------|
| 1 | A | U3004XX | 851 UPHOLSTERED TOP | 1 |
| 1 | B | 222700 | TABLE FRAME 850/851/855 | 2 |
| 1 | B | 701010 | 1X2X18 RECT INSERT CAP/GLIDE LT.GRAY | 4 |
| 1 | B | 566519 | SCREW-MACH 1/4-20 X 2 1/2 PAN PHILLIP M/S ZINC | 4 |
| 1 | B | 566528 | SCREW-TAP 10 X 2 3/4 PAN PHIL T/S ZINC | 4 |
| 2 | | 219100K | FACE PAPER HLDR ASSM. KIT | 1 |
| 2 | B | 219100 | FACE PAPER HLDR COMPLETE | 1 |
| 2 | B | 566540 | SCREW-TAP 6 X 1 BUGLE PHILLIP DRYWALL SCREW | 2 |
| 3 | | 219120K | FACE SLOT TUBE ASSM. KIT | 1 |
| 3A | B | 219120 | FACE SLOT TUBE | 1 |
| 3B | B | 566526 | SCREW-TAP 10 X 1 3/4 PAN PHIL T/S ZINC | 2 |
| 3B | B | 566315 | WASHER-1/4 NYLON SPACER (.257 X .5 X 1.0) | 2 |
| 4 | | 219165K | 851 FACE PAPER CUTTER ASSM. KIT | 1 |
| 4 | B | 219160 | FACE PAPER CUTTER | 1 |
| 4 | B | 705600 | DOUBLE SIDED FOAM TAPE, 3/4" X 36YDS | 8 in. |

NOTES

| Note | Description |
|------|--|
| A | XX- Use color code for specific part number i.e. 02 - Royal Blue |
| B | These parts are included with the UPHOLSTERED TOP |

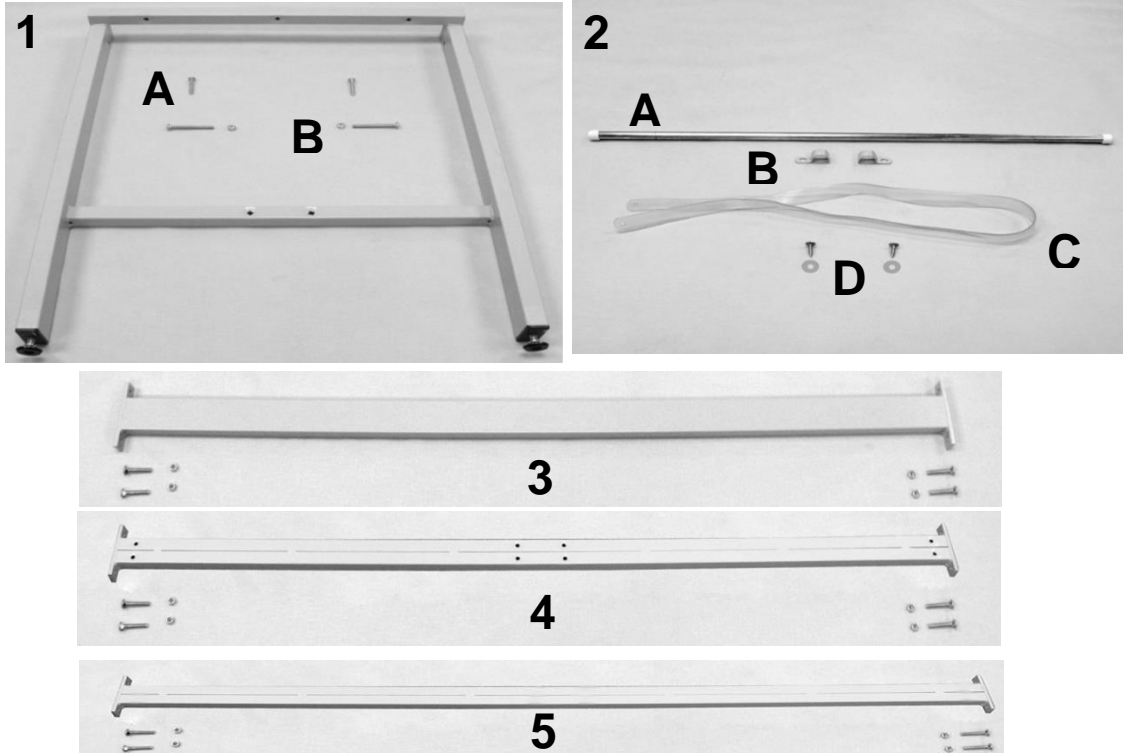

| PARTS LIST | | | | |
|--------------|---|----------------|--|-----------------|
| Image Number | Note | Part Number | Description | QTY per Product |
| 1 | | 222708K | WELDED LEG ASSM. KIT 19" | 2 |
| 1 | A | 222708 | WELDED LEG ASSM. 19" | 1 |
| 1 | | 222709K | WELDED LEG ASSM. KIT 25" (850 style) | 2 |
| 1 | A | 222709 | WELDED LEG ASSM. 25" (850 style) | 1 |
| 1 | | 222710K | WELDED LEG ASSM. KIT 30" (850 style) | 2 |
| 1 | A | 222710 | WELDED LEG ASSM. 30" (850 style) | 1 |
| 1 | | 222711K | WELDED LEG ASSM. KIT 36" (850 style) | 2 |
| 1 | A | 222711 | WELDED LEG ASSM. 36" (850 style) | 1 |
| 1 | A | 500350 | GLIDE NON-SWIVEL ZINC 5/16X1" NP LN | 2 |
| 1 | A | 501065 | 1"x2" METAL GLIDE INSERT;18 GAUGE;5/16-18 THREAD | 2 |
| 1A | A | 566506 | SCREW-MACH 1/4-20 X 1 1/2 RND PHIL M/S ZINC | 2 |
| 1B | A | 566519 | SCREW-MACH 1/4-20 X 2 1/2 PAN PHILLIP M/S ZINC | 2 |
| 1B | A | 566204 | NUT-1/4-20 HEX NYLON LOCKNUT ZINC | 2 |
| 2A | | 222705K | PAPER ROD HOLDER ASSM. KIT (850 STYLE) | 1 |
| 2A | A | 222705 | 850/851/855/856 ROD, PAPER HOLDER 1/2" ALUM | 1 |
| 2A | A | 700650 | 1/2" PLASTIC CAP, WHITE | 2 |
| 2B | | 201810 | PAPER ROD CLAMP, GRAY | 2 |
| 2C | | 271500 | VINYL TABLE STRAP | 1 |
| 2D | | 566302 | WASHER-#10 SAE F/W ZINC | 2 |
| 2D | | 566534 | SCREW-TAP 10 X 3/4 RND WASHER SQ T/S ZINC | 2 |
| 3 | | 222720 | 850/851/855/856 WELDED BRACE ASSM. | 1 |
| 3 | | 566401 | BOLT-CARRIAGE 1/4-20 X 1 1/2 ZINC | 2 |
| 3 | | 566204 | NUT-1/4-20 HEX NYLON LOCKNUT ZINC | 2 |
| 4 | B | 222730 | WELDED, 850-851-856 REINF. BRACE ASSM WELDED | 1 |
| 4 | | 566401 | BOLT-CARRIAGE 1/4-20 X 1 1/2 ZINC | 2 |
| 4 | | 566204 | NUT-1/4-20 HEX NYLON LOCKNUT ZINC | 2 |
| 5 | C | 222731 | WELDED, 850-851-856 REINF. BRACE ASSM WELDED | 1 |
| 5 | | 566401 | BOLT-CARRIAGE 1/4-20 X 1 1/2 ZINC | 2 |
| 5 | | 566204 | NUT-1/4-20 HEX NYLON LOCKNUT ZINC | 2 |
| NOTES | | | | |
| Note | Description | | | |
| A | These parts are included in ALL above listed kit. | | | |
| B | For tables with optional cabinet, or drawer. Must order double-sided tape listed on Page 3 | | | |
| C | For tables with optional shelf. Must order double-sided tape listed on Page 3 | | | |



| PARTS LIST | | | | |
|--------------|---|----------------|--|-----------------|
| Image Number | Note | Part Number | Description | QTY per Product |
| 1 | | 263960K | 850 SERIES PLAIN SHELF KIT | 1 |
| 1 | A | 263960 | 850 SERIES PLAIN SHELF | 1 |
| N/A | A | 705600 | DOUBLE SIDED FOAM TAPE, 3/4" X 36YDS | 1 ft. |
| 2 | | 263950 | 850,851,856 MODULE SHELF | 1 |
| 3 | | 266780K | 850/851/856 CABINET MODULE ASSM. KIT | 1 |
| 3A | A | 266780 | 850/851/856 CABINET MODULE | 1 |
| 3A | A | 566519 | SCREW-MACH 1/4-20 X 2 1/2 PAN PHILLIP M/S ZINC | 4 |
| 3A | A | 566204 | NUT-1/4-20 HEX NYLON LOCKNUT ZINC | 4 |
| 4 | | 266730K | 850/851/856 DRAWER MODULE KIT | 1 |
| 4 | A | 266730 | 850/851/856 DRAWER MODULE | 1 |
| 4B | A | 566519 | SCREW-MACH 1/4-20 X 2 1/2 PAN PHILLIP M/S ZINC | 4 |
| 4B | A | 566204 | NUT-1/4-20 HEX NYLON LOCKNUT ZINC | 4 |
| 4A | A | 266760K | 850/851/856 MODULE DRAWER ASSM. KIT | 1 |
| 4A | A | 266760 | 850/851/856 MODULE DRAWER ASSM | 1 |
| 4B | A | 566519 | SCREW-MACH 1/4-20 X 2 1/2 PAN PHILLIP M/S ZINC | 4 |
| 4B | A | 566204 | NUT-1/4-20 HEX NYLON LOCKNUT ZINC | 4 |
| NOTES | | | | |
| Note | Description | | | |
| A | These parts are included in the above listed kit. | | | |



| | | |
|--------|--------------------------------|--------------------------|
| Title: | 851 Treatment Table Parts List | Page 4 of 4 |
| | PL-851 | Effective Date: 11/27/17 |
| | | Rev. 01 |

Revision History

| REV | Effective Date | CCR Number | Description of Change |
|-----|----------------|------------|-----------------------|
| 01 | 11/27/17 | DCC 17-071 | Misc. minor changes. |