

8500/8510 Treatment Table with Cabinets and/or Drawer Modules



CUSTOMER INSTRUCTIONS

PLEASE READ AND FAMILIARIZE YOURSELF WITH ALL INSTRUCTIONS BEFORE USING THIS PRODUCT. If you have trouble understanding these instructions contact your dealer or Winco customer support, (800) 237-3377 before attempting to use this product; otherwise injury may occur.

Winco assumes no responsibility for damage or injury caused by improper assembly, installation, use, or maintenance of these products.



1. READ AND FOLLOW ALL DIRECTIONS.
2. NEVER stand on table or INJURY MAY OCCUR.
3. NEVER allow user to sleep unattended on table or INJURY MAY OCCUR.
4. NEVER leave an unconscious user unattended on table or INJURY MAY OCCUR.
5. DO NOT use table for transporting user in or with ANY type of vehicle or trailer. Winco tables have not been tested or approved for use by an occupant in any type of vehicle or trailer.
6. NEVER leave an infant or child unattended on table or INJURY MAY OCCUR.
7. DO NOT put hands, feet or clothing into any openings on table. Attendant MUST confirm that users hands and feet are safe or INJURY MAY RESULT.
8. Periodically, recheck tightness of all nuts, bolts and screws.
9. USE ONLY WINCO AUTHORIZED REPLACEMENT PARTS.
10. NEVER EXCEED the recommended **weight capacity of 400 lbs.** (181.4 kg).
11. SAVE THESE INSTRUCTIONS for future reference and training.

8500/8510 Weight Capacity = 400 lbs. (181.4 kg.)

Tools needed:

- Phillips-head screwdriver
- ½" wrench or ½" socket & ratchet wrench
- 7/16" wrench (multiple size wrench included)
- Scissors or box cutter
- -If using powered tools always start hardware together by hand first-

PLEASE READ AND FAMILIARIZE YOURSELF WITH ALL INSTRUCTIONS BEFORE PROCEEDING WITH ASSEMBLY

SEE "Parts Guide & List" (pg.3) for visual reference during assembly.

1. After checking your product for any shipping damage, remove all packing material and hardware that was secured for shipping.
2. Use a ½" wrench or socket, and remove the four (4) bolts that are attaching the table to the skid. (Bolts go through the plastic **Threaded Inserts** in the ends of the **Table Legs**, and the shipping skid.
3. Using at least two (2) people, lift the table off of the skid.
4. Install the included **Glide Feet** into the **Threaded Inserts**, located in bottom of **Table Legs**. (see note)

NOTE: Turning table on side or upside-down will ease this installation; however, drawers should be removed first. Please protect your **Upholstered Top** with cardboard, carpeted floor, etc... when performing this operation. Use of two or more people is HIGHLY recommended.

5. With table in the upright position, adjust the four (4) **Adjustable Glide Feet** to level the table.
6. Use a Phillips bit screwdriver and a 7/16" wrench to check all screws, nuts, and bolts to ensure they are tight and did not loosen during shipping.
7. Load paper roll onto **Paper Holder Rod**. Rod must be inserted through **Paper Rod Clamps**.


Do not over tighten nuts and bolts. Periodically check all fasteners for tightness.

Vinyl Strap Adjustment – Should the vinyl strap on your Winco table need to be adjusted, unscrew the screw and washer that currently holds either side of it in place. Pierce another hole in the strap to give desired tension and re-insert screws.

You may move the strap to a new location by removing the screws in ends of the vinyl strap and re-installing in the new location. (It may be necessary to drill 1/16" pilot holes in the wood table bottom to ease re-installation of screws.)

Wrinkles in Vinyl – Storage and shipping can cause wrinkles of the vinyl on your Winco table. This is normal. Please allow 1-2 weeks at room temperature (approx. 70 degrees) to allow for vinyl to expand back to original appearance.

Parts Guide & List

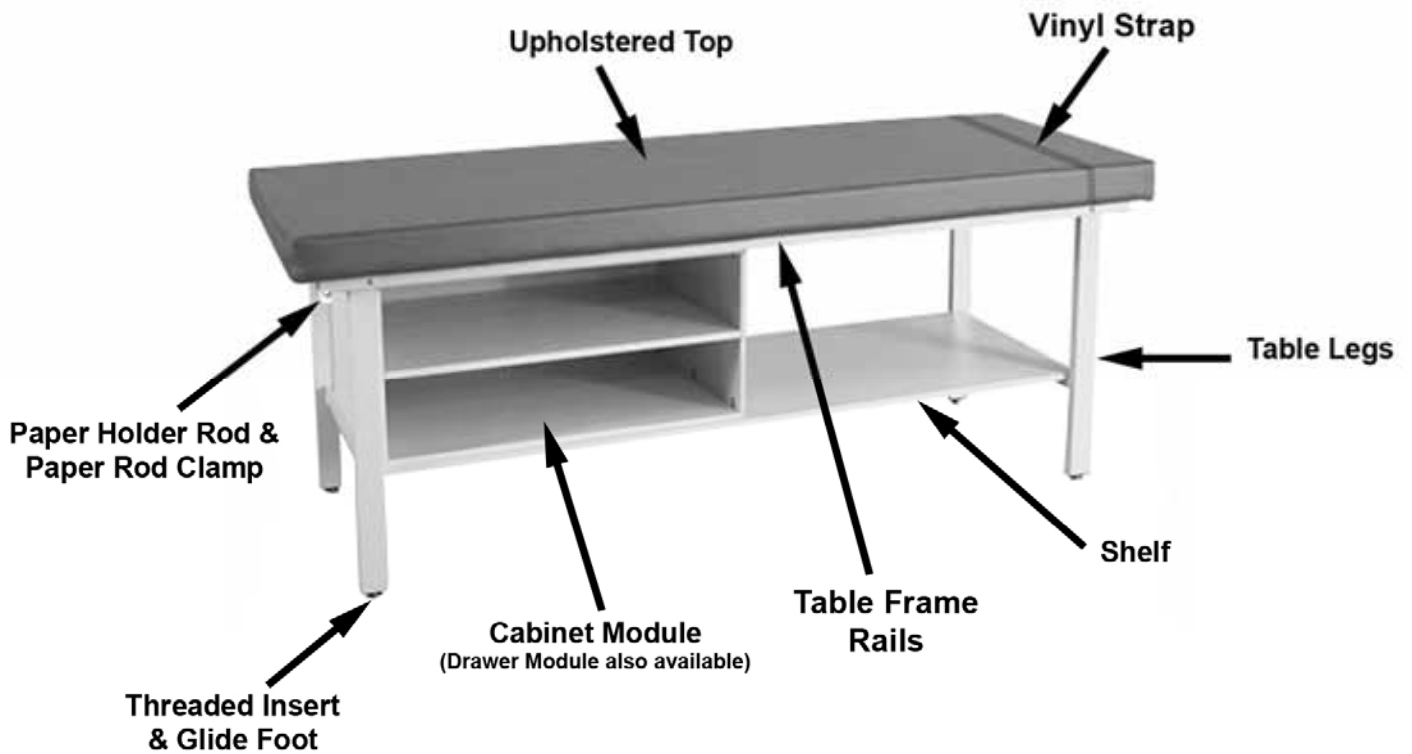

Parts List:

<u>Parts List:</u>	<u>Qty.</u>	<u>Part Number</u>
Table Frame Rails	2	222800
Table Legs	2	222710
Cross-brace	1	222730
Gray Insert Caps	4	701010
Paper Rod Clamp	2	501810
Vinyl Strap (for paper)	1	271500
Threaded Inserts (black)	4	500105
Upholstered Top	1	Call for Part Numbers

8510 Items

Face Slot Tube	1	219120
Face Paper Cutter	1	219160
Face Paper Holder	1	219100

Optional, D-Modules (drawers):

Drawer Module		266730
Module Drawer Assembly		266760
Drawer Pull		403630
8500 Series Module Shelf	1	263950

Optional, C-Modules (cabinets):

Cabinet Module		266780
8500 Series Module Shelf	1	263950

Items included with table (not installed):

Glide Feet (leveling feet)	4	500350
Paper Holder Rod	1	222705



LIMITED WARRANTY

• 5516 SW 1st Lane • Ocala, FL 34474
 • Phone (352) 854-2929 • (800) 237-3377
 • Fax (352) 854-9544
 • www.wincomfg.com
 • customerservice@wincomfg.com

TERMS

Winco Mfg., LLC. warrants this product to be free of manufacturer's defects in material and workmanship, provided that the product is used according to normal operating conditions and proper maintenance intended by manufacturer. This warranty is available only to the original retail purchaser, is non-transferable and commences on the date of retail sale; proof of purchase required.

Limited One Year Complete Product Warranty: Winco Mfg., LLC warrants the complete product for one (1) year. At Winco Mfg., LLC. sole discretion, it may repair or replace any components freight free that are found to be defective during the first year. *Winco Mfg., LLC. shall not be liable for any labor, or any other costs incurred as a result of or in conjunction with a warranty claim.*

Limited Three Year Warranty: Winco Mfg., LLC warrants the recliner mechanisms, electrical components, vinyl [from cracking or delaminating] pressurized gas springs, when new, for a period of three (3) years. At Winco Mfg., LLC. sole discretion, it may repair or replace components found to be defective. *Winco Mfg., LLC. shall not be liable for any labor, shipping or any other costs incurred as a result of or in conjunction with a warranty claim.*

Limited Lifetime Chair Frame Warranty: Winco Mfg., LLC warrants the steel chair frame, when new, for the lifetime of the original purchaser. This limited warranty does not apply to paint/finish or any components attached to the frame such as; upholstery, foam, casters, mechanisms or related parts that are covered under above warranties. *Winco Mfg., LLC shall not be liable for any labor, shipping or any other costs incurred as a result of or in conjunction with a warranty claim.*

The purchaser's exclusive remedy under this warranty shall be limited to such repair or replacement of defective components at Winco Mfg., LLC sole discretion. For warranty service, contact the dealer that the product was originally purchased from or Winco Mfg., LLC directly.

EXCLUSIONS

There are no other warranties, conditions, representations or guarantees, express or implied, made or intended by Winco Mfg., LLC and all other warranties, conditions, representations or guarantees including any warranties, conditions, representations or guarantees under any Sale of Good Act or Like legislation or statute is hereby expressly excluded. Any and all other implied warranties shall not extend beyond the duration of the express warranty. Liability for incidental or consequential damages is excluded to the extent permitted by law. Some states do not allow the exclusion or limitation so the above limitation or exclusion may not apply to you. This warranty gives you specific legal rights. In addition you may also have other rights, which vary from state to state.

GENERAL PROVISIONS

No warranty herein contained or set out shall apply when damage or repair is caused by any of the following:

1. Damage in transit.
2. Accident, alteration, abuse or misuse of product ie. exceeding weight capacities, applying inordinate pressure to footrest/leg rest, use in Mental Health facilities or like institutions.
3. Fire, water damage, theft, war, riot, hostility, acts of God.

Examples contained in this list are not to be construed as all-inclusive.

