

3170 Designer Three Panel Screen

3180 Healing Vistas Privacy Screen



CUSTOMER INSTRUCTIONS

PLEASE READ AND FAMILIARIZE YOURSELF WITH ALL INSTRUCTIONS BEFORE USING THIS PRODUCT.
If you have trouble understanding these instructions contact your dealer or Winco customer support, (800) 237-3377,
e-mail: customerservice@wincomfg.com before attempting to use this product; otherwise injury may occur.

Winco assumes no responsibility for damage or injury caused by the improper assembly, installation or use of these products.



1. READ AND FOLLOW ALL DIRECTIONS.
2. NEVER place screen on a cart, table, stand or other objects; Screen could fall and INJURY MAY OCCUR.
3. NEVER place screen near open flame or over a radiator or heat register.
4. NEVER place screen in front of a door or exit sign; or during emergency screen could hinder evacuation.
5. SUFFOCATION DANGER to children if vinyl is not attached to screen; Screen could block nose and mouth and prevent breathing. VINYL SCREEN IS NOT A TOY.
6. DO NOT allow children to play with screen; Screen could fall and INJURY MAY OCCUR.
7. Screen should ALWAYS be positioned on level surface.
8. NEVER stand or ride on screen legs; INJURY MAY OCCUR.
9. DO NOT use screen in or with ANY type of vehicle or trailer.
10. DO NOT attach objects to screen frame; Screen can tip over and INJURY MAY OCCUR.
11. Remove from service; Any screen with broken or missing parts, torn vinyl, or other visible damage.
12. USE ONLY WINCO AUTHORIZED REPLACEMENT PARTS.
13. SAVE THESE INSTRUCTIONS for future reference and training.

Tools needed:

- Scissors or box cutter
- 5/32 Allen Wrench (provided)

PLEASE READ AND FAMILIARIZE YOURSELF WITH ALL INSTRUCTIONS BEFORE PROCEEDING WITH ASSEMBLY

1. After checking your product for any shipping damage, remove all packing material and hardware that was secured for shipping. Cut tape and/or plastic with scissors, being careful not to damage screen.

LEG INSTALLATION SEE PHOTOS, NEXT PAGE

2. Insert both **Leg Frame Assemblies (#3)** into the bottom of the **Wing/Mid Frame Assembly (#1A)**.
Line up the holes in the **(#3)** w/ the holes in the **(#1A)**.
3. Using the **Allen Wrench** provided screw in **1-Button Socket Screw (3D)**, per leg, through **(#1A)** and into **(#3)**



PANEL REPLACEMENT SEE PHOTOS, NEXT PAGE

4. To replace **Vinyl Panels (#4)**, hold the screen by the top **Screen Rod (#2A)**.
5. Next, using the **Allen Wrench** provided, push in on the small hole just below the **Snap Button (#2B)**.
Re-install the **(#2A)** by repeating the above procedure, then release the **(#2B)** when the **(#2A)** is in place

CARE & MAINTENANCE OF YOUR SCREEN

6. Periodically check all fasteners for tightness.
7. To clean **Vinyl Panels** use only mild liquid soap and water.

Wrinkles in Vinyl – Storage and/or shipping causes wrinkles in the vinyl of your Winco Screen; This is normal. Please allow 1-2 weeks at room temperature (approx. 70+ degrees) to allow for vinyl to return to original appearance.

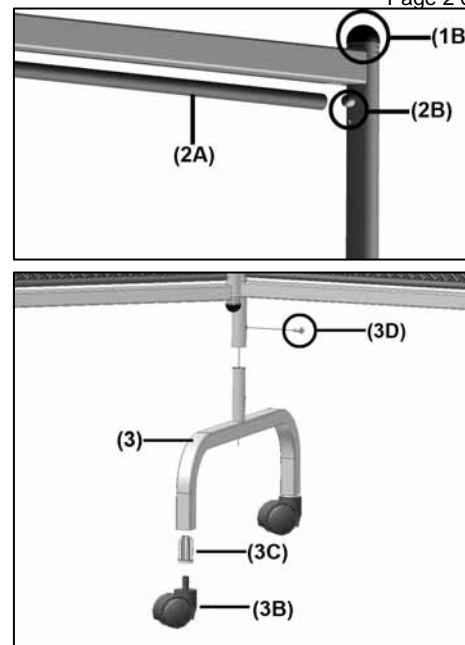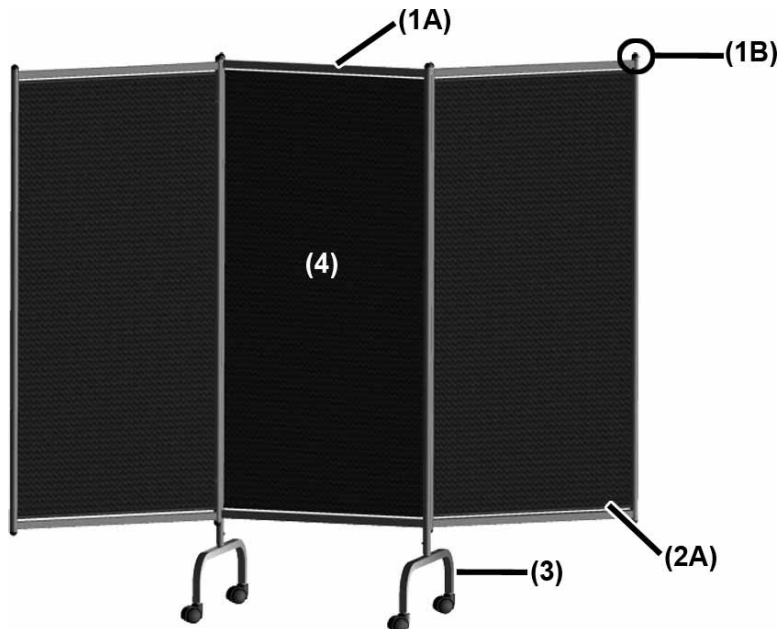


Items Included with Screen:

Qty.

Sizing chart – Part number

(1A) Wing/Mid Frame Assembly Kit	1	215620
Mid Frame	1	<i>Available in kit ONLY</i>
Wing Frame	1	<i>Available in kit ONLY</i>
Aluminum Hinge	4	<i>Available in kit ONLY</i>
(1B) Domed Plastic Insert	10	700845
(2A) Screen Rod	6	215616
(2B) Snap Buttons	4	401800
(3) Leg Frame Assembly	2	215603
(3B) 2" Castor	4	400000
(3C) Castor Insert	4	702200
(3D) 3/4" Button Socket Screw	2	566572
5/32" Allen Wrench (<i>not shown</i>)	1	566014
(4) 3170 Vinyl Panel	3	Call for part number
(5) 3180 Vinyl Panels (SEE BELOW)	1 (3-panel SET)	909100
(5A) Left-side Panel*	1	909110
(5B) Middle Panel	1	909120
(5C) Right-side Panel*	1	909130

**3180 Left/Right Panels are determined as if looking at the scene side of screen*

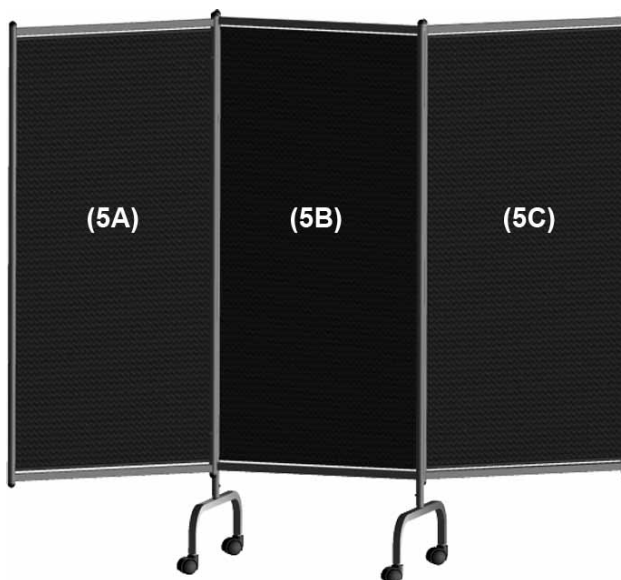



LIMITED WARRANTY

5516 SW 1st Lane • Ocala, FL 34474
 Phone (352) 854-2929 • (800) 237-3377 • Fax (352)
 854-9544

TERMS

Winco Mfg., LLC. warrants this product to be free of manufacturer's defects in material and workmanship, provided that the product is used according to normal operating conditions and proper maintenance intended by manufacturer. This warranty is available only to the original retail purchaser, is non-transferable and commences on the date of retail sale; proof of purchase required.

Limited One Year Complete Product Warranty: Winco Mfg., LLC warrants the complete product for one (1) year. At Winco Mfg., LLC. sole discretion, it may repair or replace any components freight free that are found to be defective during the first year. *Winco Mfg., LLC. shall not be liable for any labor, or any other costs incurred as a result of or in conjunction with a warranty claim.*

Limited Three Year Warranty: Winco Mfg., LLC warrants the recliner mechanisms, electrical components, vinyl [from cracking or delaminating] pressurized gas springs, when new, for a period of three (3) years. At Winco Mfg., LLC. sole discretion, it may repair or replace components found to be defective. *Winco Mfg., LLC. shall not be liable for any labor, shipping or any other costs incurred as a result of or in conjunction with a warranty claim.*

Limited Lifetime Chair Frame Warranty: Winco Mfg., LLC warrants the steel chair frame, when new, for the lifetime of the original purchaser. This limited warranty does not apply to paint/finish or any components attached to the frame such as; upholstery, foam, casters, mechanisms or related parts that are covered under above warranties. *Winco Mfg., LLC shall not be liable for any labor, shipping or any other costs incurred as a result of or in conjunction with a warranty claim.*

The purchaser's exclusive remedy under this warranty shall be limited to such repair or replacement of defective components at Winco Mfg., LLC sole discretion. For warranty service, contact the dealer that the product was originally purchased from or Winco Mfg., LLC directly.

EXCLUSIONS

There are no other warranties, conditions, representations or guarantees, express or implied, made or intended by Winco Mfg., LLC and all other warranties, conditions, representations or guarantees including any warranties, conditions, representations or guarantees under any Sale of Good Act or Like legislation or statute is hereby expressly excluded. Any and all other implied warranties shall not extend beyond the duration of the express warranty. Liability for incidental or consequential damages is excluded to the extent permitted by law. Some states do not allow the exclusion or limitation so the above limitation or exclusion may not apply to you. This warranty gives you specific legal rights. In addition you may also have other rights, which vary from state to state.

GENERAL PROVISIONS

No warranty herein contained or set out shall apply when damage or repair is caused by any of the following:

1. Damage in transit.
2. Accident, alteration, abuse or misuse of product ie. exceeding weight capacities, applying inordinate pressure to footrest/leg rest, use in Mental Health facilities or like institutions.
3. Fire, water damage, theft, war, riot, hostility, acts of God.



Examples contained in this list are not to be construed as all-inclusive.