



Field Installation and Usage Instructions

Accessory Part No. TMA91-15
Surgical Bar Extension Set for Leg Section

***For Use with SP-007-02 Leg Section
and TMM-1254-11 Surgical Bars***

5516 S.W. 1st Lane – Ocala, FL 34474-9307
Phone: 1-800-237-3377 - Fax: 1-352-854-9544
www.transmotionmedical.com

Document No.: IM TMA91-15
Revision: B

TABLE OF CONTENTS

TABLE OF CONTENTS.....	2
IMPORTANT NOTES ABOUT THIS MANUAL.....	3
SPECIAL NOTES - SIGNAL WORDS	3
TOOLS REQUIRED.....	4
PARTS INCLUDED	4
INSTALLING SURGICAL BAR EXTENSION SET.....	4
CAUTIONARY USAGE INSTRUCTIONS.....	6
CLEANING INSTRUCTIONS	6
PREVENTIVE MAINTENANCE	7

IMPORTANT NOTES ABOUT THIS MANUAL

This Field Installation and Usage Instruction manual is intended to assist you with installation of your TransMotion Medical Product. Carefully read this manual before using the equipment or doing service / maintenance on it. If you are unable to understand the WARNINGS, CAUTIONS, and instructions, contact TransMotion Medical Customer Service before attempting to operate or service the equipment. Otherwise, personal injury or property damage may result.

To ensure safe operation of this device, it is essential that methods and procedures be established for educating and training staff on the safe and effective operation of the product.

TransMotion Medical reserves the right to change specifications without notice.

The information contained in this document is subject to change without notice.

Specifications listed are nominal, and operation may vary slightly from unit to unit due to tolerance variations.

Figures contained within this document may not depict exactly the model you have. This document may cover many closely related models.

SPECIAL NOTES - SIGNAL WORDS

Signal words are used in this manual and apply to hazards which could result in injury or property damage. The following is a definition of those signal words as used in this document:

DANGER

Indicates an imminently hazardous situation which, if not avoided, will result in death or serious injury.

WARNING

Indicates a potentially hazardous situation which, if not avoided, could result in death or serious injury.

CAUTION

Indicates a potentially hazardous situation which, if not avoided, may result in minor or moderate injury. It may also be used to alert against unsafe practices or potential property damage hazards.

NOTICE

Provides important information, makes special instructions clearer, or provides service personnel information to make maintenance easier.

TOOLS REQUIRED

Installation tools are not required for this accessory.

PARTS INCLUDED

Verify all parts listed have been received:

- TMA91-15 (Surgical Bar Extension Set for Leg Section)

If not, contact TMM Service Department toll-free at 1-800-237-3377.



INSTALLING SURGICAL BAR EXTENSION SET



NOTICE

Installation instructions are valid for both sides of leg section.


Extension for (patient's) left side is shown here. Some parts are transparent for clarity.

Installation Steps:




ENSURE FINGERS ARE NOT PINCHED DURING INSTALLATION.

- 1) Position surgical bar extension by placing "V" shape formed by pin and extension end around mounting boss located near seat.

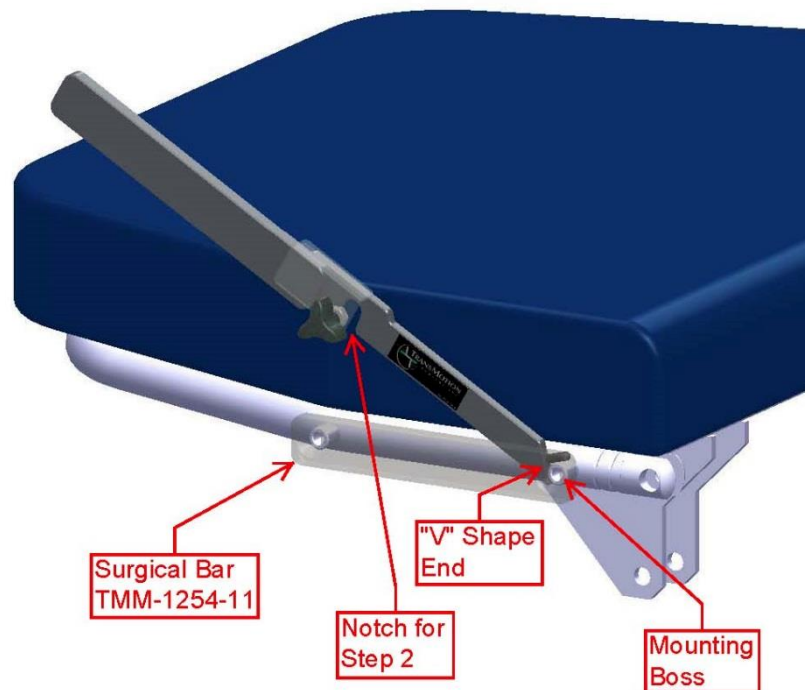

Installation Steps (continued):

- 2) Rotate surgical bar extension downward, and place notch over mounting boss located near foot end.
- 3) Tighten knob clockwise by hand to secure extension in place.
- 4) Repeat installation steps for other side of leg section.

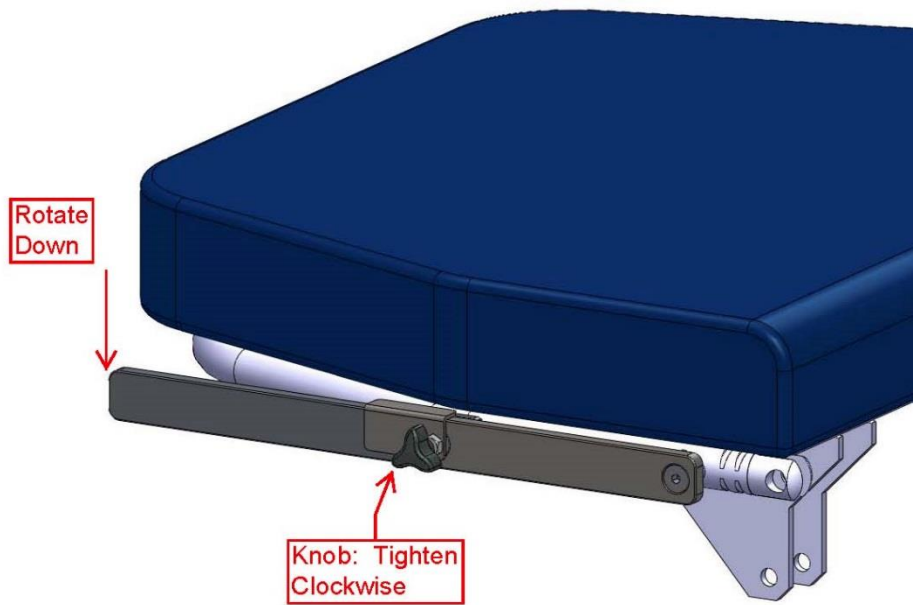

The surgical bar extensions are ready for use!



CAUTIONARY USAGE INSTRUCTIONS

When using this accessory, please follow these cautionary usage instructions:

CAUTION ENSURE STABILITY OF BOTH EXTENSIONS AND ANY ATTACHED EQUIPMENT PRIOR TO EVERY MEDICAL PROCEDURE TO PREVENT EQUIPMENT SLIPPAGE.

CAUTION REMOVE EXTENSIONS PRIOR TO PATIENT EGRESS TO PREVENT PATIENT TRIPPING.

CAUTION REMOVE EXTENSIONS PRIOR TO CHAIR TRANSPORT TO PREVENT PERSONAL INJURY AND PROPERTY DAMAGE.

CLEANING INSTRUCTIONS

CAUTION STEAM AND/OR PRESSURE CLEANING CHAIR WILL VOID WARRANTY

COMPONENT	FREQUENCY	PROCEDURE	CLEANING AGENT *	COMMENT
All Rails and Painted Metal	As required/in accordance with facility policies	Wipe with damp cloth and allow to air dry	Facility approved detergents, disinfectants & water	Check for chipped paint/chrome
Base Cover	As required/in accordance with facility policies	Wipe with damp cloth and allow to air dry	Facility approved detergents, disinfectants & water	Check for peeling or missing labels
Actuators and Battery Pack	As required/in accordance with facility policies	Wipe with damp cloth and allow to air dry	Facility approved detergents, disinfectants & water	Clean exterior surfaces only with minimal water
Pendants	As required/in accordance with facility policies	Wipe with damp cloth and allow to air dry	Facility approved detergents, disinfectants & water	Clean exterior surfaces only with minimal water
All Other Surfaces	As required/in accordance with facility policies	Wipe with damp cloth and allow to air dry	Facility approved detergents, disinfectants & water	Check for damage
TransMotion Medical Accessories	As required/in accordance with facility policies	Wipe with damp cloth and allow to air dry	Facility approved detergents, disinfectants & water	Check for damage
3rd Party Accessories	As required/in accordance with facility policies	Consult manual provided by accessory manufacturer.		

* Use of Facility approved detergents and disinfectants must comply with the instructions provided by the manufacturer(s) of those products.

Upholstery Care & Cleaning Instructions

Proper care is essential in ensuring the durability and reliability of TransMotion Medical upholstery. In general, all products should be:

1. Cleaned
2. Disinfected (in accordance with facility policy)
3. Rinsed
4. Allowed to air dry

It is important to note that the terms “disinfecting” and “cleaning” should not be used interchangeably. Disinfectants alone will not provide adequate cleaning since they do not have the appropriate properties to cut grease or oil and remove grime, hair or skin oils. The appearance, feel and performance of your upholstery may diminish if not cleaned properly.

Cleaning:

Remove hair, grime and body oils. Your upholstery should be cleaned on a regular basis with a damp cloth soaked in a mild soap and water solution. Avoid harsh detergents or chemicals that could damage the finish of your product. If disinfecting with manufacturer approved chemicals or bleach, the upholstery MUST be wiped off using clean water on a damp cloth and allowed to air-dry as a final step. Failure to rinse upholstery with clean water can result in a build-up of residues that, over time, may lead to drying, cracking or other undesirable changes to appearance, feel and performance.

Retain all instructional tags for future use.

Contact the manufacturer of the fabric used in your upholstery for a current list of approved disinfectants. TransMotion Medical Customer Care can assist with identifying which upholstery was used with your Stretcher-Chair.

For customer supplied and non-standard materials, please refer to the individual manufacturer’s cleaning instructions.

PREVENTIVE MAINTENANCE

To ensure proper operation, this accessory should be maintained as described below.

COMPONENT	FREQUENCY	PROCEDURE	TOOL	COMMENT
Retention Knob	Every three months	<ul style="list-style-type: none">• Check for ease of knob rotation.	-	If difficult to turn, accessory may need replaced.
Pin and Cap	Every three months	<ul style="list-style-type: none">• Check that pin and cap are present.• Check that cap is not damaged or torn.	-	Extensions may be used without pin and cap, but fit may be loose.