
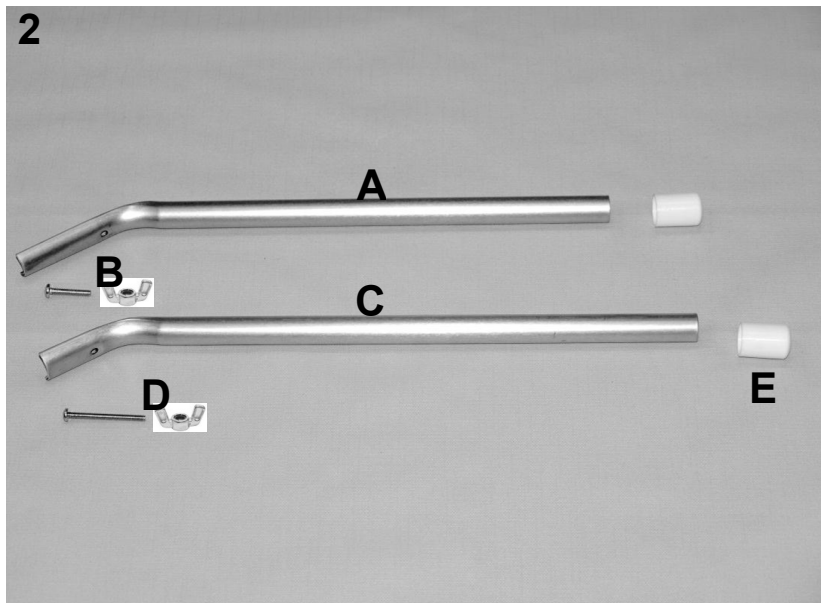


3730-3740


Privess™ Modular Privacy Screen

PARTS LIST

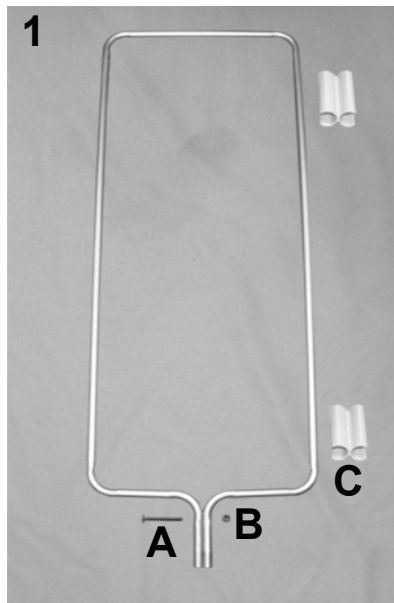

| PARTS LIST | | | | |
|--------------|------|----------------|----------------------------------------------|-----------------|
| Image Number | Note | Part Number | Description | QTY per Product |
| 1 | | 216640K | 373-374 SINGLE SCREEN FRAME ASSM. KIT | 3 |
| 1 | | 216640 | SCREEN FRAME (373 STYLE) | 1 |
| 1A | | 566598 | 10-24 x 1 3/4" CARRIAGE BOLT | 1 |
| 1B | | 566203 | NUT-10-24 HEX NYLON LOCK NUT ZINC | 1 |
| 1C | | 704550 | PLASTIC SCREEN HINGE | 2 per frame |
| 2 | | 216633K | LEG SET W/ HARDWARE 373 KIT | 1 |
| 2A | | 216630 | MID LEG (363,373 style) With Notch | 2 |
| 2B | | 566414 | BOLT- CARRIAGE 10-24 X 1 1/4 ZINC | 2 |
| 2C | | 216635 | WING LEG (363, 373 STYLE) Without Notch | 4 |
| 2D | | 566413 | BOLT- CARRIAGE 10-24 X 2 ZINC | 2 |
| 2B,2D | | 566222 | 10-24 NYLON-LOCK WING NUT | 4 |
| 2E | | 700800 | PLASTIC ENDCAP 3/4" WHITE | 6 |
| 2 | | 216634K | LEG SET W/ HARDWARE 374 KIT | 1 |
| 2A | | 216630 | MID LEG (363,373 style) With Notch | 4 |
| 2B | | 566414 | BOLT- CARRIAGE 10-24 X 1 1/4 ZINC | 4 |
| 2C | | 216635 | WING LEG (363, 373 STYLE) Without Notch | 4 |
| 2D | | 566413 | BOLT- CARRIAGE 10-24 X 2 ZINC | 2 |
| 2B,2D | | 566222 | 10-24 NYLON-LOCK WING NUT | 6 |
| 2E | | 700800 | PLASTIC ENDCAP 3/4" WHITE | 8 |

NOTE: Contents of this document are subject to change without notice. Please refer to our website to be sure you are utilizing the most current Rev.

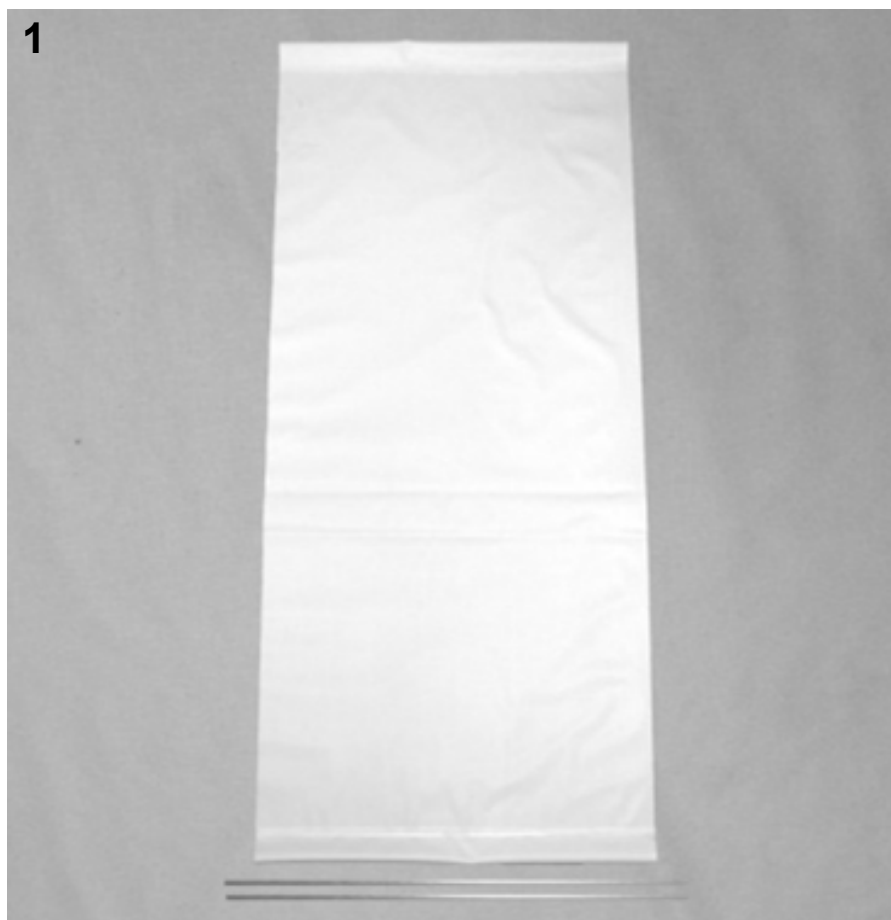

| PARTS LIST | | | | |
|--------------|----------------------------------------------------------------------------------|-------------|-------------------------------------|-----------------|
| Image Number | Note | Part Number | Description | QTY per Product |
| 1 | A | 299500 | 373, 374 VINYL PANEL STANDARD WHITE | SEE NOTES |
| 1 | A,B | 2995XX | 373, 374 SURE CHEK PANEL | SEE NOTES |
| 1 | C | 216660 | SCREEN ROD (373 STYLE) | SEE NOTES |
| NOTES | | | | |
| Note | Description | | | |
| A | 373= (3) Panels, 374= (4) Panels | | | |
| B | XX- For SURECHEK use color code for specific part number i.e. 93 - Mint (299593) | | | |
| C | Qty. (2) per panel | | | |

Rev. 01 – Initial document created.

NOTE: Contents of this document are subject to change without notice. Please refer to our website to be sure you are utilizing the most current Rev.