
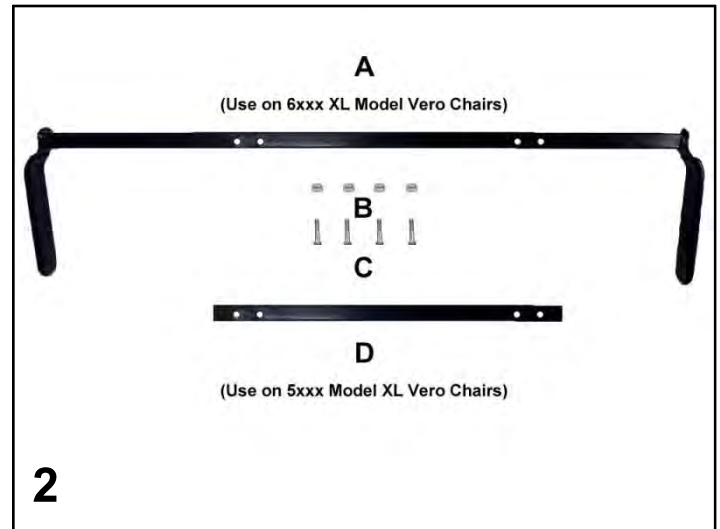



Vero Power Recline Chair

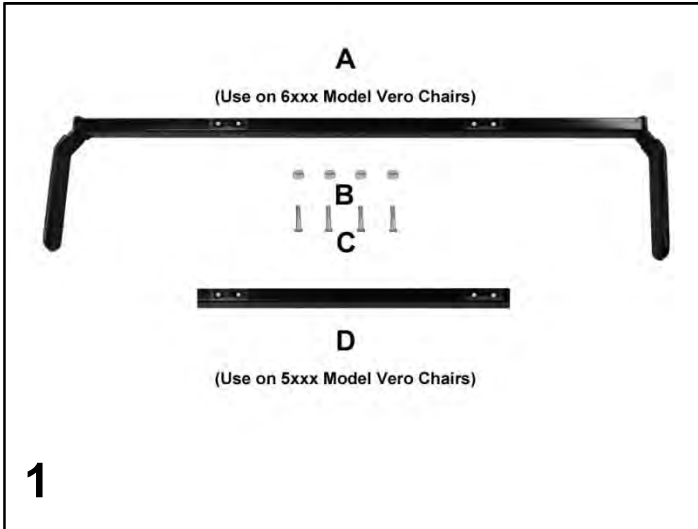

PARTS LIST


Image Number	Note	Part Number	Description	QTY per Product
Standard Chair				
1	A	207062BK	Trendelenberg Pedal – Vero Kit	1
1A	B	407062	Trendelenberg Pedal - VERO	1
1B	B	566204	NUT-1/4-20 HEX NYLON LOCKNUT ZINC	4
1C	B	566583	SCREW, HEX CAP 1/4-20 X 1 1/2" GRADE 5, ZINC	4
1	C	206012BK	Non-Trend. Tube W/O Pedal Kit	1
1D	B	206012B	Non-Trend. Tube W/O Pedal	1
1B	B	566204	NUT-1/4-20 HEX NYLON LOCKNUT ZINC	4
1C	B	566583	SCREW, HEX CAP 1/4-20 X 1 1/2" GRADE 5, ZINC	4
XL Chair				
2	D	207072BK	Trendelenberg Pedal – Vero XL Kit	1
2A	B	407072	Trendelenberg Pedal - VERO XL	1
2B	B	566204	NUT-1/4-20 HEX NYLON LOCKNUT ZINC	4
2C	B	566583	SCREW, HEX CAP 1/4-20 X 1 1/2" GRADE 5, ZINC	4
2	E	206022BK	Non-Trend. Tube W/O Pedal – XL Kit	1
2D	B	206022B	Non-Trend. Tube W/O Pedal - XL	1
2B	B	566204	NUT-1/4-20 HEX NYLON LOCKNUT ZINC	4
2C	B	566583	SCREW, HEX CAP 1/4-20 X 1 1/2" GRADE 5, ZINC	4
Note	Description			
A	Used on chair models - 6P0x, 6P4x			
B	Part is included in above listed kits - Parts can be ordered separately in certain cases.			
C	Used on chair models - 5P0x, 5P4x			
D	Used on chair models - 6P1x, 6P5x			
E	Used on chair models - 5P1x, 5P5x			

NOTE: Contents of this document are subject to change without notice. Please refer to our website to be sure you are utilizing the most current Rev.

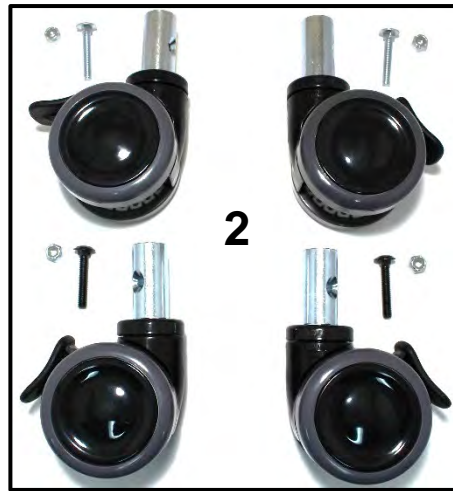


PARTS LIST				
Image Number	Note	Part Number	Description	QTY per Product
Standard Push Back Style Chair				
1, 2		566400	BOLT-CARRIAGE ZINC 1/4-20 X 1 1/4	2
1, 2		566399	BOLT- CARRIAGE BLACK ZINC 1/4-20 X 1 1/4	2
1, 2		566204	NUT-1/4-20 HEX NYLON LOCKNUT ZINC	4
HM-HT Push Back Style Chair				
1, 2		566399	BOLT-CARRIAGE BLACK ZINC 1/4-20 X 1 1/4	2
1, 2		566401	BOLT-CARRIAGE 1/4-20 X 1 1/2 ZINC	2
1, 2		566204	NUT-1/4-20 HEX NYLON LOCKNUT ZINC	4
2	A	400400	3" TOTAL LOCK, TWIN WHEEL CASTER	4
1	B	400620	CASTER, NYLON 5" RIGID, BLACK	2
1	C	400625	CASTER, NYLON 5" TOTAL LOCK W/PRECISION BEARING WHEEL, BLACK	2
3	D	566330	BLACK FLAT WASHER	4
3	D	700500	PEDESTAL FOOT, PLASTIC	4
3	D	702207	CASTER SOCKET INSERT HD, 1" SQ 16-18 GA	4
NOTES				
Note	Description			
A	Replacements ONLY for chairs with 3" Casters (Not compatible with Footplate)			
B	Replacement for 5" Front Casters.			
C	Replacement for 5" Rear Casters.			
D	All parts (IMAGE 3) MUST be ordered to switch to Pedestal Foot option. (Not compatible with Footplate)			
<i>Left & Right is determined - as if sitting in the chair</i>				

NOTE: Contents of this document are subject to change without notice. Please refer to our website to be sure you are utilizing the most current Rev.

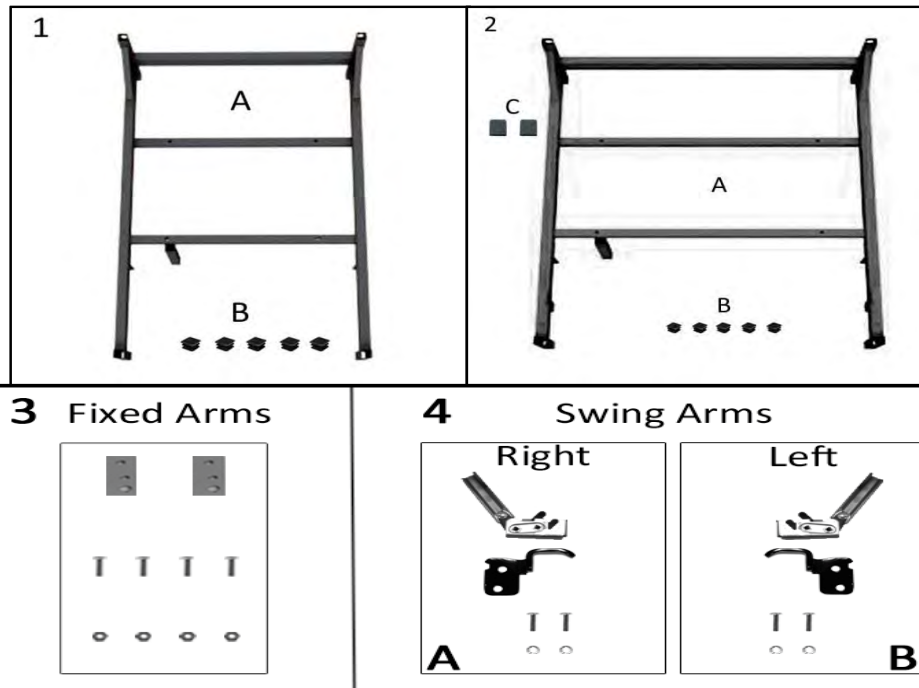

PARTS LIST				
Image Number	Note	Part Number	Description	QTY per Product
1A	A	206000B	WELDED MAIN FRAME (VERO)	1
1B, 2B		700700	SQ 1" RIBBED CAP, BLACK	5
2A		206010B	WELDED MAIN FRAME (VERO XL)	1
2C		700890	CAP, FLAT RECTANGULAR, GRAY, .25" X 1"	2
3	B		Fixed Arm Style Chair	
3		202830B	LATCH PLATE, FIXED ARM, BLACK	2
3		566572	SCREW, BUTTON HEAD 1/4-20 X 3/4", ZINC	4
3		566204	NUT-1/4-20 HEX NYLON LOCKNUT ZINC	4
4	C		Swing Arm Style Chair	
4A		202820K	ROTARY LATCH RIGHT (624) KIT	1
4A		402855	ROTARY LATCH, HD SPRING (RIGHT)	1
4A		302100	ROTARY LATCH HANDLE (624 STYLE)	1
4A		510200	RIVET 1/4 X 3/8 X 7/16 HEAD	1
4A		206155RB	SUPPORT BRKT, UNDER-ARM, RIGHT, BLACK (VERO)	1
4A		566592	1/4-20 X .88 SOCKET BUTTON HEAD CAP SCREW, ZINC	2
4A		566301	WASHER-1/4 INT. TOOTH LOCKWASHER ZINC	2
4B		202825K	ROTARY LATCH LEFT (624) KIT	1
4B		402850	ROTARY LATCH, HD SPRING (LEFT)	1
4B		302100	ROTARY LATCH HANDLE (624 STYLE)	1
4B		510200	RIVET 1/4 X 3/8 X 7/16 HEAD	1
4B		206155LB	SUPPORT BRKT, UNDER-ARM, LEFT, BLACK (VERO)	1
4B		566592	1/4-20 X .88 SOCKET BUTTON HEAD CAP SCREW, ZINC	2
4B		566301	WASHER-1/4 INT. TOOTH LOCKWASHER ZINC	2
NOTES				
Note	Description			
A	Please contact Winco Customer Service/Tech Support (800-237-3377) for assistance with MAIN FRAME.			
B	Fixed Arm Style Chairs are models 5P0x, 5P1x, 6P0x and 6P1x.			
C	Swing Arm Style Chairs are models 5P4x, 5P5x, 6P4x and 6P5x.			

NOTE: Contents of this document are subject to change without notice. Please refer to our website to be sure you are utilizing the most current Rev.

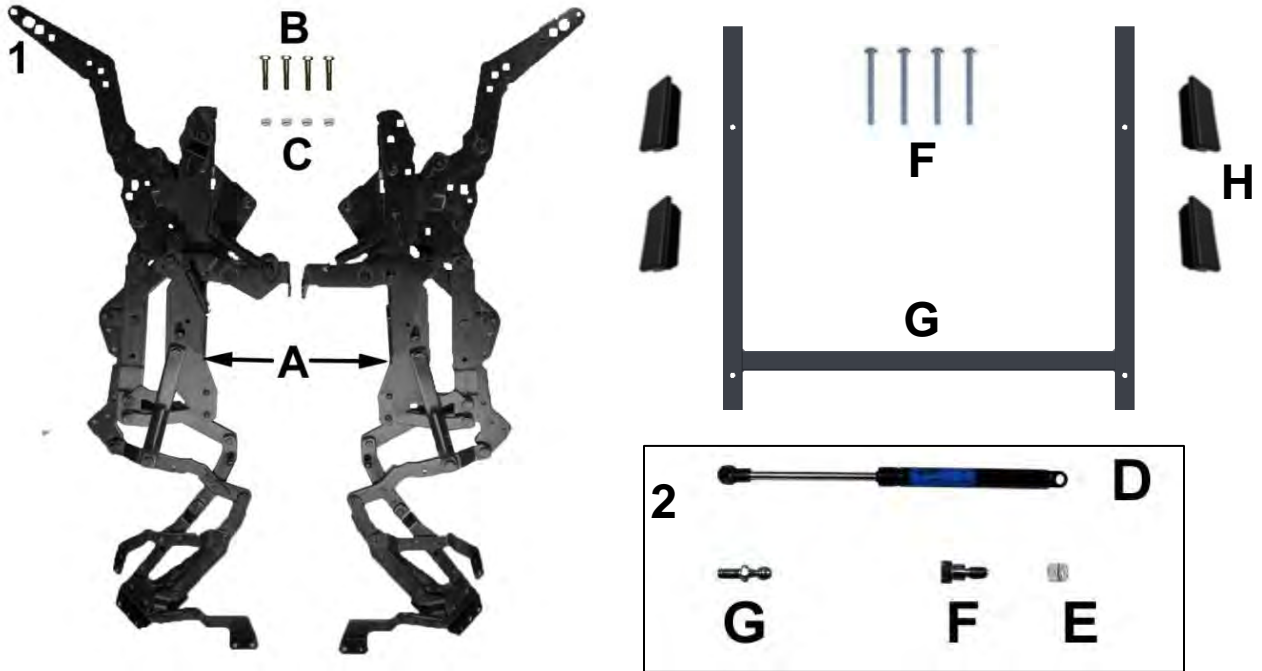


PARTS LIST

Image Number	Note	Part Number	Description	QTY per Product
1, 2	A		Push Back Style Chair	
1		205080K	STD Vero Seat Mech Replacement Kit	1
1A	B	407080L	POWER MECHANISM VERO (LEFT)	1
1A	B	407080R	POWER MECHANISM VERO (RIGHT)	1
1B	B	407400	REAR CROSS WELDMENT, POWER MECH STD	1
1C	B	407405	FRONT CROSS ASSY, POWER MECH STD	1
1D	B	566583	1/4-20 X 1.75" HEX CAP SCREW, GRADE 8, ZINC	4
1E	B	566204	NUT-1/4-20 HEX NYLON LOCKNUT ZINC	4
1F	B	566560	10-24 X 2.75" RD PHIL M/S ZINC.	4
1G	B	209105B	WELDED SEAT RAIL, VERO POWER STD	1
1H	B	701010	1X2X18 RECT INSERT CAP/GLIDE, BLACK	4
1		205081K	XL Vero Seat Mech Replacement Kit	1
1A	B	407080L	POWER MECHANISM VERO (LEFT)	1
1A	B	407080R	POWER MECHANISM VERO (RIGHT)	1
1B	B	407500	REAR CROSS WELDMENT, POWER MECH XL	1
1C	B	407505	FRONT CROSS ASSY, POWER MECH XL	1
1D	B	566583	1/4-20 X 1.75" HEX CAP SCREW, GRADE 8, ZINC	4
1E	B	566204	NUT-1/4-20 HEX NYLON LOCKNUT ZINC	4
1F	B	566560	10-24 X 2.75" RD PHIL M/S ZINC.	4
1G	B	209205	WELDED SEAT RAIL VERO POWER XL	1
1H	B	701010	1X2X18 RECT INSERT CAP/GLIDE, BLACK	4
2			Damper	
2		205656K	Damper Replacement Kit	1
2D	B	405655	DAMPER, BI-DIRECTIONAL	1
2E	B	566204	NUT-1/4-20 HEX NYLON LOCKNUT ZINC	1
2F	B	566615	5/16" Shoulder x 1/4" Shoulder Length x 1/4"-20 18-8 Stainless Steel Socket Shoulder Bolt	1
2G	B	504640	5/16-18 BALL STUD	1
NOTES				
Note	Description			
A	Push Back Style Chairs are models 5Yxx and 6Yxx.			
B	Part is included in above listed kits - Part may be ordered separately in some cases.			
C	All parts listed in above images 1 & 2 are included in kit.			


NOTE: Contents of this document are subject to change without notice. Please refer to our website to be sure you are utilizing the most current Rev.



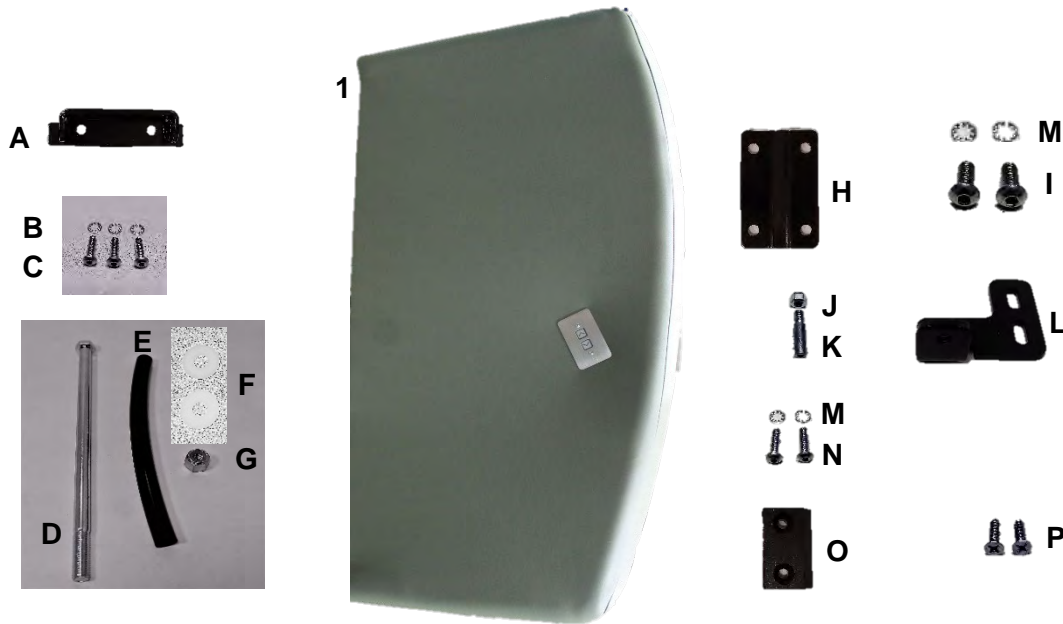


PARTS LIST				
Image Number	Note	Part Number	Description	QTY per Product
1			Fixed Arm Style Chair - Version 2	
See Notes	A, E, F	U0421XX	UPHOLSTERED FIXED-ARM PANEL LEFT - VERO (WITHOUT POWER SWITCH)	1
1		U0429XX	UPHOLSTERED FIXED-ARM PANEL LEFT - VERO (WITH POWER SWITCH)	1
1	A, E	U0529XX	UPHOLSTERED FIXED-ARM PANEL RIGHT - VERSION 2	1
1			Swing Arm Style Chair - Version 2	
See Notes	A, E, F	U0522XX	UPHOLSTERED SWING-ARM PANEL LEFT - VERO (WITHOUT POWER SWITCH)	1
1	A, E	U0430XX	UPHOLSTERED SWING-ARM PANEL LEFT - VERO (WITH POWER SWITCH)	1
1	A, E	U0530XX	UPHOLSTERED SWING-ARM PANEL RIGHT - VERSION 2	1
1A	B, D, E	202178B	WELDED HINGE LONG, MAIN FRAME 180 BLACK (VERO) (VERSION 2)	1
1B	B, D, E	566301	WASHER-1/4 INT. TOOTH LOCKWASHER ZINC (VERSION 2)	3
1C	B, D, E	566572	SCREW, BUTTON HEAD 1/4-20 X 3/4", ZINC	3
1D	B, D, E	566602	3/8-16 X 9" HEX CAP SCREW, GRADE 5, ZINC SAE J429 (VERSION 2)	1
1E	B, D, E	260110	TUBING, NYLON BOLT SLEEVE (VERSION 2)	1
1F	B, D, E	566325	WASHER, NYLON .375 ID X 1" X .062 THICK (VERSION 2)	2
1G	B, D, E	566217	NUT, 3/8-16 HEX NYLON LOCKNUT ZINC (VERSION 2)	1
1H	B, C	206175B	ARM MOUNTING PLATE, STATIONARY (VERO) - BLACK	1
1I	B	566592	1/4-20 X .88 SOCKET BUTTON HEAD CAP SCREW, ZINC	2
1J	B	566205	NUT-5/16-18 HEX NYLON LOCK NUT ZINC	1
1K	B	202770	STRIKER PIN, 3/8 X 1" (5/16-18 THREAD)	1
1L	B	202746LB	Striker Bracket, Arm (VERO) - Black -Left	1
1L	B	202746RB	Striker Bracket, Arm (VERO) - Black -Right	1
1M	B	566301	WASHER-1/4 INT. TOOTH LOCKWASHER ZINC	4
1N	B	566572	SCREW, BUTTON HEAD 1/4-20 X 3/4", ZINC	2
1O	B, D	700620	ARM SUPPORT BLOCK, NYLON	1
1P	B, D	C-WF-ZP-14-10	SCREW, WD PH FL HD ZP #14 THD x 1.000 LG	2
Note	Description			
A	XX - Use color code for specific part number i.e. 02 - Royal Blue			
B	Part is included in above listed kits - Part can be ordered separately. Quantities are for each arm kit.			
C	Used on Fixed-Arm Style Chair only!			
D	Used on Swing-Arm Style Chair only!			
E	Parts listed under "Version 2" should be used on all Vero model chairs manufactured on or after 02/08/18.			
F	See PL-5Xxx-6Xxx and PL-5Yxx-6Yxx for left side upholstered arm images WITHOUT Power Switch.			
OPTIONS	For options on replacement arms see page 13			
	<i>Left & Right is determined - as if sitting in the chair</i>			

NOTE: Contents of this document are subject to change without notice. Please refer to our website to be sure you are utilizing the most current Rev.

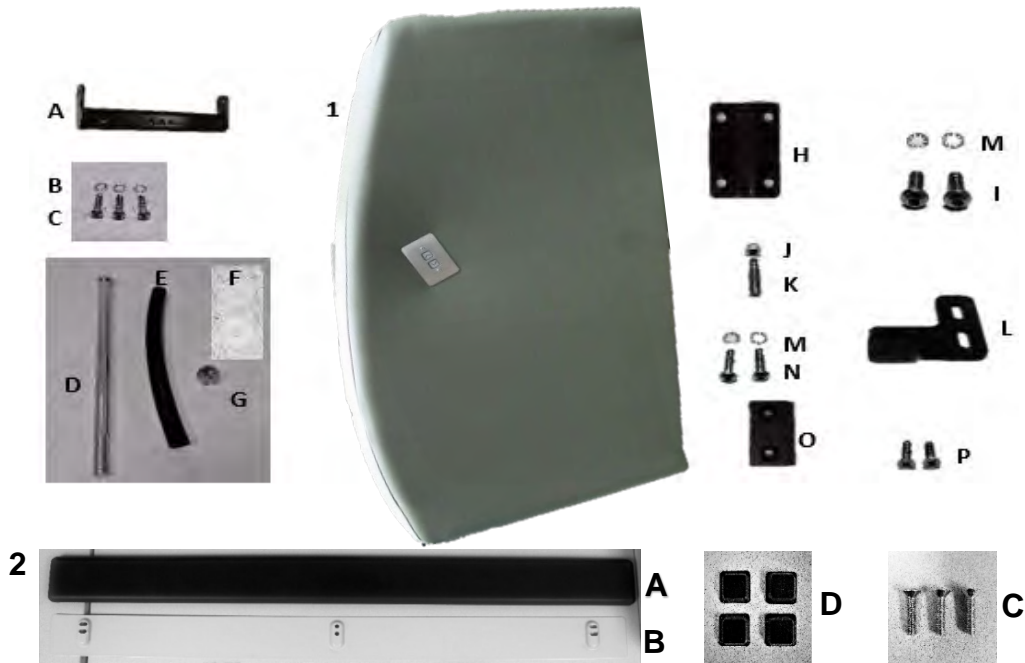


PARTS LIST				
Image Number	Note	Part Number	Description	QTY per Product
1			Fixed Arm Style Chair with Urethane Arm	
1	A	U0427XX	UPHOLSTERED FIXED-ARM PANEL LEFT (WITHOUT POWER SWITCH)	1
1	A	U0531XX	UPHOLSTERED FIXED-ARM PANEL LEFT (WITH POWER SWITCH)	1
1	A	U0527XX	UPHOLSTERED FIXED-ARM PANEL RIGHT	1
			Swing Arm Style Chair with Urethane Arm	
1	A	U0428XX	UPHOLSTERED SWING-ARM PANEL LEFT (WITHOUT POWER SWITCH)	1
1	A	U0532XX	UPHOLSTERED SWING-ARM PANEL LEFT (WITH POWER SWITCH)	1
1	A	U0528XX	UPHOLSTERED SWING-ARM PANEL RIGHT	1
1A	B, D	202178B	WELDED HINGE LONG, MAIN FRAME 180 BLACK (VERO)	1
1B	B, D	566301	WASHER-1/4 INT. TOOTH LOCKWASHER ZINC	3
1C	B, D	566572	SCREW, BUTTON HEAD 1/4-20 X 3/4", ZINC	3
1D	B, D	566602	3/8-16 X 9" HEX CAP SCREW, GRADE 5, ZINC SAE J429	1
1E	B, D	260110	TUBING, NYLON BOLT SLEEVE	1
1F	B, D	566325	WASHER, NYLON .375 ID X 1" X .062 THICK	2
1G	B, D	566217	NUT, 3/8-16 HEX NYLON LOCKNUT ZINC	1
1H	B, C	206175B	ARM MOUNTING PLATE, STATIONARY (VERO) - BLACK	1
1I	B	566592	1/4-20 X .88 SOCKET BUTTON HEAD CAP SCREW, ZINC	2
1J	B	566205	NUT-5/16-18 HEX NYLON LOCK NUT ZINC	1
1K	B	202770	STRIKER PIN, 3/8 X 1" (5/16-18 THREAD)	1
1L	B	202746LB	Striker Bracket, Arm (Gen II) - Black -Left	1
1L	B	202746RB	Striker Bracket, Arm (Gen II) - Black -Right	1
1M	B	566301	WASHER-1/4 INT. TOOTH LOCKWASHER ZINC	4
1N	B	566572	SCREW, BUTTON HEAD 1/4-20 X 3/4", ZINC	2
1O	B, D	700620	ARM SUPPORT BLOCK, NYLON	1
1P	B, D	SC-WF-ZP-14-100	SCREW, WD PH FL HD ZP #14 THD x 1.000 LG	2
2A	B,E	708040	Urethane Arm Cover	1
2B	B,E	203692B	Mounting Plate, Steel	1
2D	B,E	701900	Sq Bumper,.078 w/ Adhesive, Black	4
2C	B,E	566607	1/4-20 X 1" Flat Mach Screw, Zinc	3
Notes				
A	XX - Use color code for specific part number i.e. 02 - Royal Blue			
B	Part is included in above listed kits - Part can be ordered seperately. Quantities are for each arm kit.			
C	Used on Fixed-Arm Style Chair only!			
D	Used on Swing-Arm Style Chair only!			
E	Used on Urethane Arm Style only!			
OPTIONS	For options on replacement arms see page 13			
	Left & Right is determined - as if sitting in the chair			

NOTE: Contents of this document are subject to change without notice. Please refer to our website to be sure you are utilizing the most current Rev.

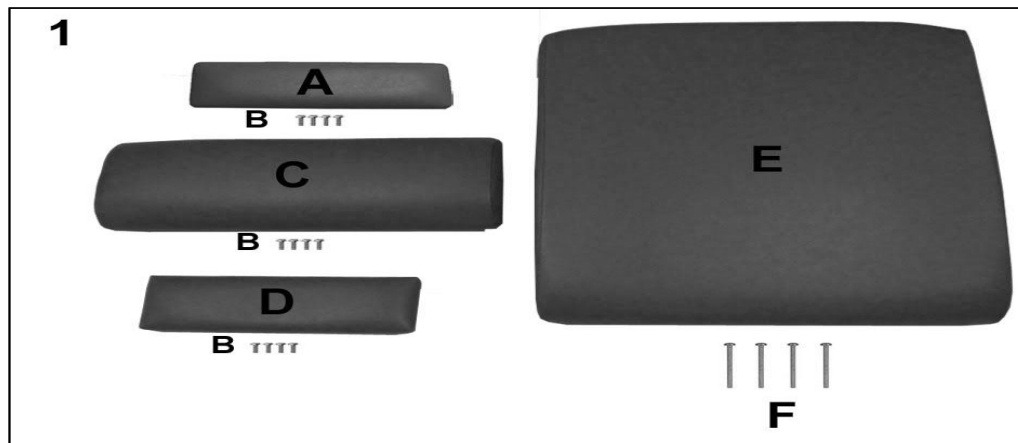

PARTS LIST				
Image Number	Note	Part Number	Description	QTY per Product
1			STANDARD VERO CHAIR	
1A	A	U1205XX	Upholstered Mid Ottoman Vero	1
1B	C	566511	SCREW-MACH 10-24 X 3/4" PAN PHIL M/S ZINC	4
1C		U1017XX	Upholstered Legrest Vero	1
1D		U1108XX	Upholstered Footrest Vero	1
1E	A,B	U0317XX	Upholstered Seat Vero	1
1F		566560	10-24 X 2.75" RD PHIL M/S ZINC.	4
1			XL VERO CHAIR	
1A	A	U1303XX	Upholstered Mid Ottoman Vero XL	1
1B	C	566511	SCREW-MACH 10-24 X 3/4" PAN PHIL M/S ZINC	4
1C		U1018XX	Upholstered Legrest Vero XL	1
1D		U1107XX	Upholstered Footrest Vero XL	1
1E	A,B	U0318XX	Upholstered Seat Vero XL	1
1F		566560	10-24 X 2.75" RD PHIL M/S ZINC.	4
NOTES				
Note	Description			
A	XX- Use color code for specific part number i.e. 02 - Royal Blue Screws needed to attach this product are included			
B	For chairs with Heat and Massage see related notes below (H6).			
C	Parts 566511 are used on Parts represented by A, C, and D.			
OPTIONS	For TB133 Add TB after Part Number (i.e.U0317XX-TB) Use 00 if option is not needed (i.e.U0317XX- 00 -H6 - For Upholstered Seat without TB with H6)			
H6	For Heat Only chairs made 12-05-18 to present add identifier after TB option i.e. U0317XX-TB-H6 For Heat & Massage chairs made 03-01-19 to present add identifier after TB option i.e. U0317XX-TB-H6			
	<i>Left & Right is determined - as if sitting in the chair</i>			

NOTE: Contents of this document are subject to change without notice. Please refer to our website to be sure you are utilizing the most current Rev.

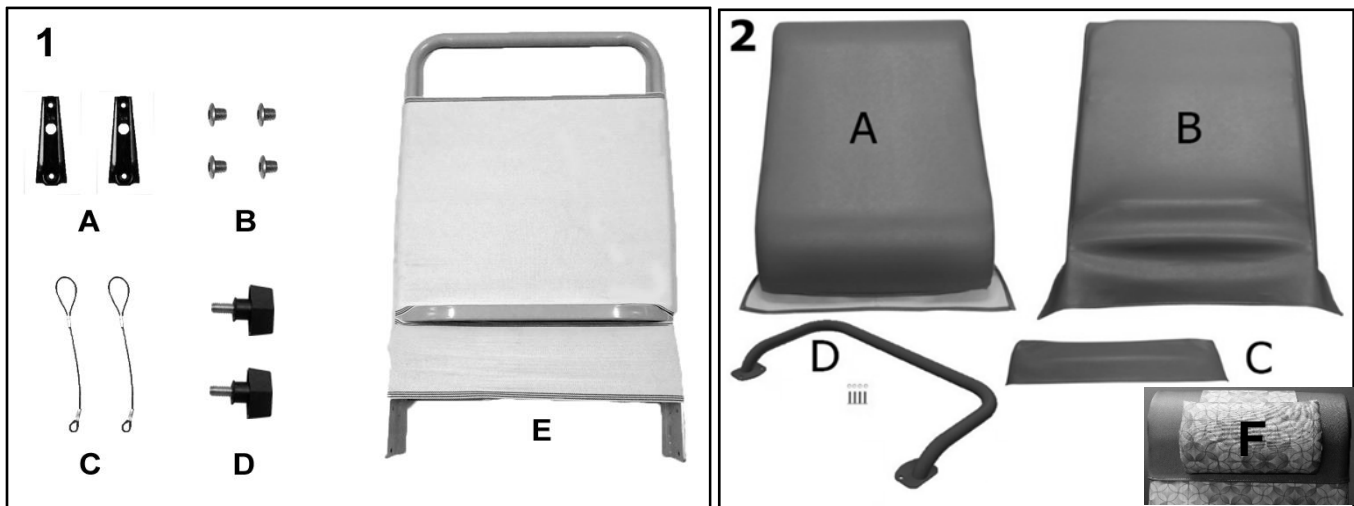

PARTS LIST

Image Number	Note	Part Number	Description	QTY per Product
Standard Vero Chair				
1,2	C,E	U0225XX	Upholstered Back w/Frame Assm Kit, Vero STD	1
1	A	206030BK	Vero Welded Back Frame Kit	1
1A	B	407019	KD SLEEVE, 624 MECH	2
1B	B	566584	SCREW, 1/4-20 x 1/4" SS BUTTON SOCKET HEAD SCREW	4
1C	B	201105	LANYARD W/SLEEVES, KNOB (Vero)	2
1D	B	501235	Knob, 2-Arm, 1_4-20 x 1/2 Stud	2
1E	B	206030B	Welded Back Frame (Vero) - Black	1
1E	B	902530	17" ULTRALITE WEBBING, CUT 26"	1
1E	B	902535	6" ULTRALITE WEBBING, CUT 26"	1
2A,2B	C,E	U0223XX	UPHOLSTERED BACK VERO	1
2D	D	206035BK	Vero Push Handle Kit STD	1
2D	D	566592	1/4-20 X .88 Socket Button Head Cap Screw, Zinc	4
2D	D	566301	WASHER 1/4 INT. TOOTH LOCKWASHER ZINC	4
2E	C	NP05-XX	HEADREST COVER VERO STD CHAIR W/ PILLOW	1
2C	C	HCVS-XX	Headrest Cover Vero	1
XL Vero Chair				
1,2	C,E	U0226XX	Upholstered Back w/Frame Assm Kit, Vero XL	1
1	A	206050BK	Vero XL Welded Back Frame Kit	1
1A	B	704014	KD Bracket (VERO)	2
1B	B	566584	SCREW, 1/4-20 x 1/4" SS BUTTON SOCKET HEAD SCREW	4
1C	B	201105	LANYARD W/SLEEVES, KNOB (Vero)	2
1D	B	501235	Knob, 2-Arm, 1_4-20 x 1/2 Stud	2
1E	B	206050B	Welded Back Frame (Vero XL) - Black	1
1E	B	902540	16" ULTRALITE WEBBING, CUT 30"	1
1E	B	902545	6" ULTRALITE WEBBING, CUT 30"	1
2A,2B	C,E	U0224XX	UPHOLSTERED BACK VERO XL	1
2D	D	206060BK	Welded Push Handle Kit (Vero XL) - Black	1
2D	D	566592	1/4-20 X .88 Socket Button Head Cap Screw, Zinc	4
2D	D	566301	WASHER 1/4 INT. TOOTH LOCKWASHER ZINC	4
2F	C	NP06-XX	HEADREST COVER VERO XL W/ PILLOW	1
2C	C	HCVS-XX	Headrest Cover Vero XL	1

See notes on next page.

NOTE: Contents of this document are subject to change without notice. Please refer to our website to be sure you are utilizing the most current Rev.

Page | 8
Copyright 2020 Winco Mfg., LLC

PL-5P-6P REV 01
Effective: 02/26/21


UPHOLSTERED BACK NOTES	
Note	Description
A	Hardware needed to attach this product is included
B	Parts are included in the above listed kits
C	XX- Use color code for specific part number i.e. 02 - Royal Blue
D	Refer to "OPTIONS" when ordering a KIT with Push Handle option
E	For chairs with Heat and Massage see related notes below (H5 or H6).
OPTIONS	For TB133 Add " TB " after Part Number i.e. U0225-XX-TB-00-00. For chairs with the Push Handle option use "PH" identifier after
Heat Only	For chairs w/ Heat Only made from 12-05-18 to present; Use H6 after TB or 00 (i.e.U0225xx-TB- H6)
Heat & Massage	For chairs w/ Heat-Massage made from 03-01-19 to present; Use M6 after TB or 00 (i.e.U0225xx-TB- M6)
	<i>Left & Right is determined - as if sitting in the chair</i>

NOTE: Contents of this document are subject to change without notice. Please refer to our website to be sure you are utilizing the most current Rev.

Page | 9
Copyright 2020 Winco Mfg., LLC

PL-5P-6P REV 01
Effective: 02/26/21



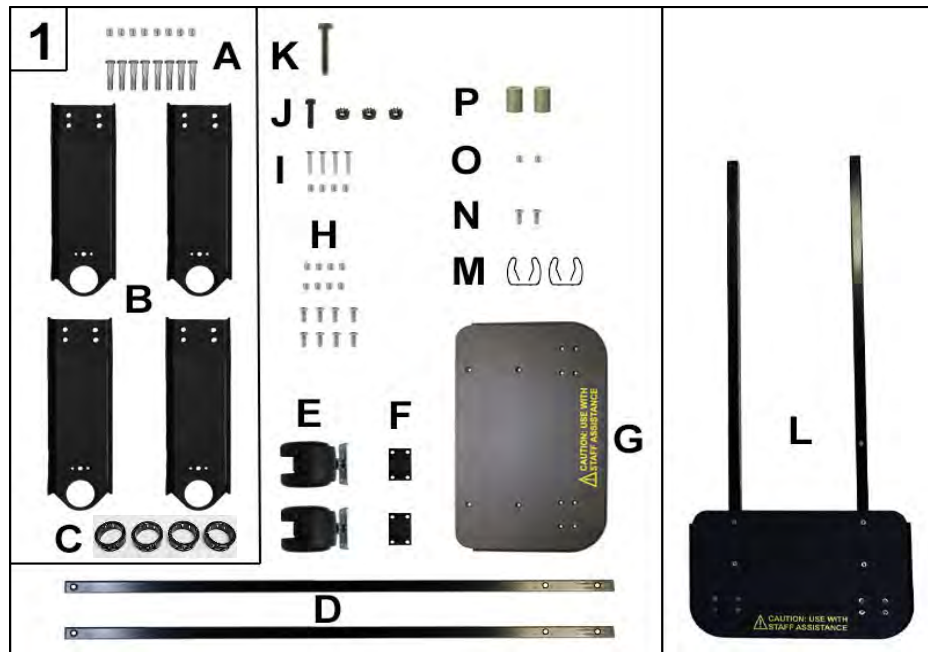


PARTS LIST

Image Number	Note	Part Number	Description	QTY per Product
1A,B,C	A	206182BK	Bracket, Foot Support Tube (Vero) Kit	1
1A	B	566583	SCREW, HEX CAP 1/4-20 X 1 1/2" GRADE 5, ZINC	8
1A	B	566204	NUT-1/4-20 HEX NYLON LOCKNUT ZINC	8
1B	B	206183B	Bracket, Foot Support Tube W/ clip Holes (Vero)	4
1C	B	700924	BUSHING, SNAP-IN, NYLON BLACK	4
1M	B	TMM-146-04	RETAINING CLIP	2
1N	B	566510	SCREW-MACH 10-24 X 1/2 PAN PHIL MACHINE SCREW	2
1O	B	566203	NUT-10-24 HEX NYLON LOCK NUT ZINC	2
1P	B	501105	5/8 RD X 1.00 X .252 ID RD SPACER NYLON	2
1L	A	206040K	Vero Footplate w/Footrest Tubes Kit	
1D	B	206021B	Footrest Tube (GenII) - Black	2
1E	B	400160	Caster, Sit-Lock, Hooded, Twin Soft Wheel w-Mount Plate (Vero)	2
1F	B	206185B	Spacer Block, Footplate Caster (Vero) - Black	2
1G	B	206040TS	Footplate w-CAUTION USE WITH STAFF ASSISTANCE graphic (Vero)	1
1H	B	566616	#10-24 X 3/4" PHILLIPS DRIVE FLAT HEAD (BLACK) GRD 18-8 SS MS	8
1H	B	566203	NUT-10-24 HEX NYLON LOCK NUT ZINC	8
1I	B	566617	#10-24 X 1-1/4" PHILLIPS DRIVE FLAT HEAD (BLACK) GRD 18-8 SS MS	4
1I	B	566203	NUT-10-24 HEX NYLON LOCK NUT ZINC	4
1J	B	566415	BOLT- HEX 1/4-20 X 1 1/4, GRADE 5, BLACK FINISH	1
1J	B	566223	1/4-20 LOCK NUT, BLACK FINISH	3
1K	B	566483	SCREW, HEX CAP 1/4-20 X 2" GRADE 5, ZINC	1
1M	B	TMM-146-04	RETAINING CLIP	2
1N	B	566510	SCREW-MACH 10-24 X 1/2 PAN PHIL MACHINE SCREW	2
1O	B	566203	NUT-10-24 HEX NYLON LOCK NUT ZINC	2
1P	B	501105	5/8 RD X 1.00 X .252 ID RD SPACER NYLON	2
1A&L	C	206040TK	Vero Footplate Complete Kit	1
NOTES				
Note	Description			
A	Kit includes ALL items listed below			
B	It is NOT recommended that these parts be sold separately however in special cases they can be.			
C	Includes all items from above kits 206182BK & 206040K			

NOTE: Contents of this document are subject to change without notice. Please refer to our website to be sure you are utilizing the most current Rev.

Page | 10
Copyright 2020 Winco Mfg., LLC

PL-5P-6P REV 01
Effective: 02/26/21

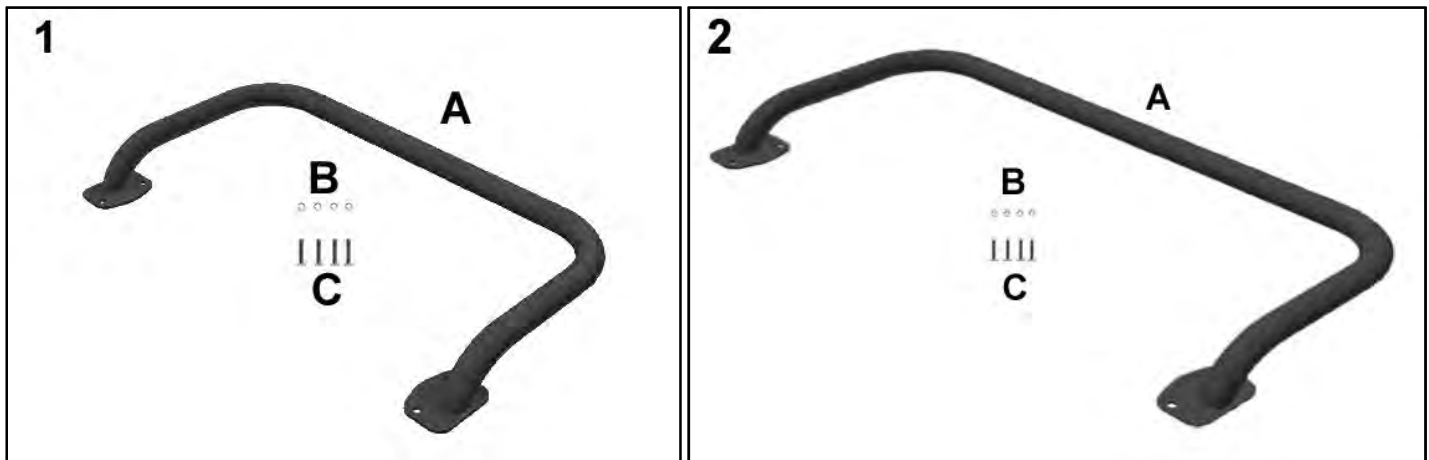


PARTS LIST

Image Number	Note	Part Number	Description	QTY per Product
1	A	206035BK	Vero Push Handle Kit	1
1A	B	206035B	Welded Push Handle	1
1B	B	566572	SCREW, BUTTON HEAD 1/4-20 X 3/4", ZINC	4
1C	B	566301	WASHER-1/4 INT. TOOTH LOCKWASHER ZINC	4
2	A	206060BK	Vero XL Push Handle Kit	1
2A	B	206060B	Welded Push Handle (Vero XL) - Black	1
2B	B	566572	SCREW, BUTTON HEAD 1/4-20 X 3/4", ZINC	4
2C	B	566301	WASHER-1/4 INT. TOOTH LOCKWASHER ZINC	4
NOTES				
Note	Description			
A	Kit includes ALL items listed below			
B	It is NOT recommended that these parts be sold separately however in special cases they can be.			

NOTE: Contents of this document are subject to change without notice. Please refer to our website to be sure you are utilizing the most current Rev.

Page | 11
Copyright 2020 Winco Mfg., LLC

PL-5P-6P REV 01
Effective: 02/26/21

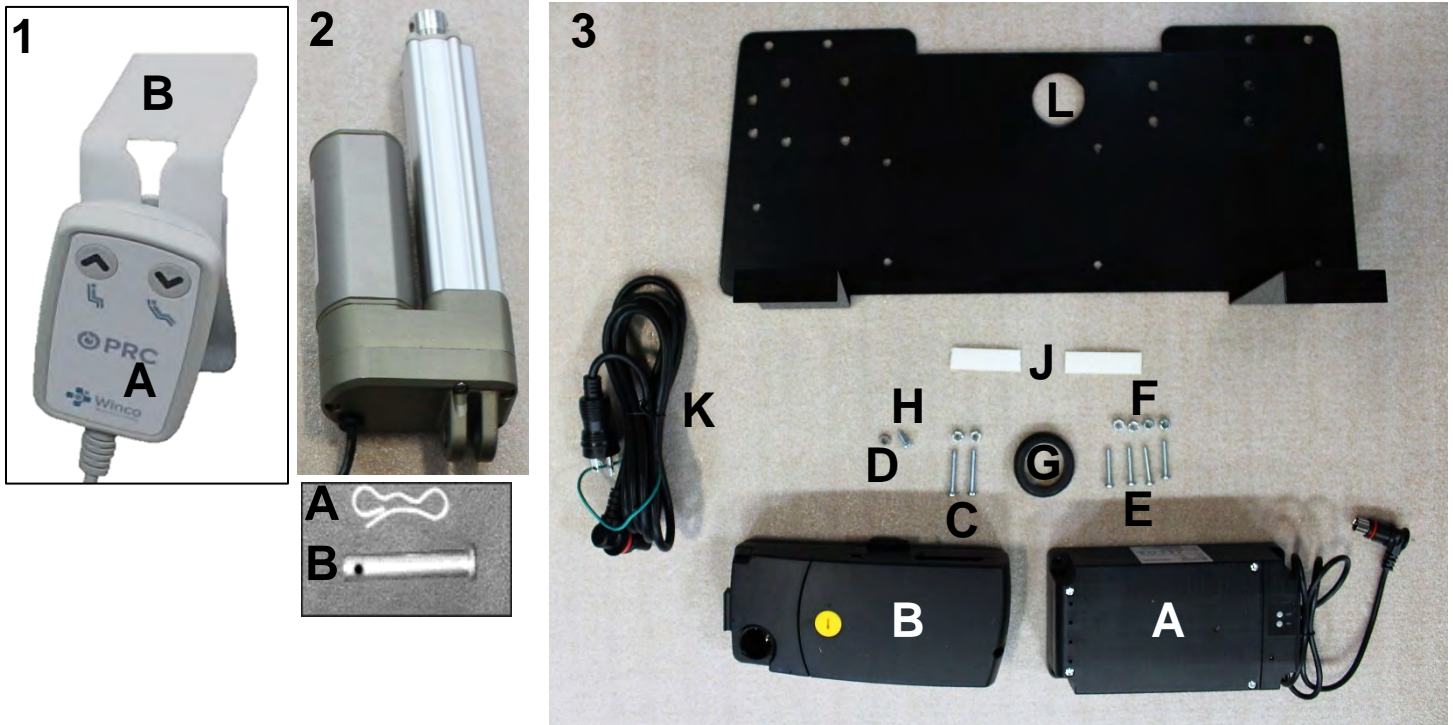

PARTS LIST				
Image Number	Note	Part Number	Description	QTY per Product
1		205655K	PENDANT WITH HOLDER KIT	1
1A		490321	PENDANT WITH OVERLAY	1
1B		490420	PRC PENDANT HOLDER	1
2		490300	ACTUATOR, POWER RECLINE, TA16	1
2A		566002	BOW TIE COTTER PIN	2
2B		506005	CLEVIS PIN, .391 X 1.5	2
3A, E, F		490315K	BATTERY ADD-ON/REPLACEMENT KIT, VERO PRC	1
3A		490315	BACK- UP BATTERY, TBB4	1
3B		490310	CONTROL BOX, TC21	1
3C		566515	SCREW-MACH 10-24 X 1 3/4 PAN PHILLIP M/S ZINC	2
3D		TMM-764-04	NUT #10-32 TOOTHED LOCK WASHER	2
3E		566513	SCREW-MACH 10-24 X 1 1/4 PAN PHILLIP M/S ZINC	4
3F		566203	NUT-10-24 HEX NYLON LOCK NUT ZINC	6
3G		501825	BLACK RUBBER PUSH-IN GROMMET 1-3/4"	1
3H		TMM-126-04	SCREW, 10-32 X 1/2 - TYPE 23	1
3J		705605	DOUBLE SIDED MOLDING TAPE	2
3K		490330	VERO, POWER CORD, TAC Y-CORD	1
3L		206700B	POWER SUPPLY, REAR MOUNTING PLATE POWDER COAT BLACK	1
Not Shown		506025	FLAME RETARDANT CABLE TIE 8"	3

NOTE: Contents of this document are subject to change without notice. Please refer to our website to be sure you are utilizing the most current Rev.

Page | 12
Copyright 2020 Winco Mfg., LLC

PL-5P-6P REV 01
Effective: 02/26/21

PARTS LIST	
Left Upholstered Arm Replacement Options	
OPTIONS	Description
Upholstered Fixed-Arm Panel	Example: U0429XX use specific color code in place of "XX" (i.e. 02 - Royal Blue).
Upholstered Swing-Arm Panel	Example: U0430XX use specific color code in place of "XX" (i.e. 02 - Royal Blue).
Place Holder	Use 00 if option is not need i.e. U0411XX- 00-00-UB
TB133	Add TB after Part Number i.e. U0411XX- TB
Rear IV Pole (Left)	Add PB as illustrated U0411XX-00-00- PB
Foley Bag Holder (left)	Add L0 as illustrated U0411XX-00-00- L0
Rear IV Pole and Foley Bag (Left)	Add LB as illustrated U0411XX-00-00- LB
Urethane Arm	Add UB as illustrated U0411XX-00-00-00- UB
Right Upholstered Arm Replacement Options	
OPTIONS	Description
Upholstered Fixed-Arm Panel	Example: U0529XX use specific color code in place of "XX" (i.e. 02 - Royal Blue).
Upholstered Swing-Arm Panel	Example: U0530XX use specific color code in place of "XX" (i.e. 02 - Royal Blue).
Place Holder	Use 00 if option is not need i.e. U0521XX- 00-H5
TB133	Add TB after Part Number i.e. U0521XX- TB
O2 Holder	Add O2 after Part Number i.e. U0521XX- O2
TB133 and O2 Holder	Add TO after Part Number i.e. U0521XX- TO
Heat Only	For chairs w/ Heat Only made 12-05-18 to present; Use H6 after TB or 00 (i.e U0511XX-TB- H6)
Heat and Massage	For chairs w/ Heat-Mass made 03-01-19 to present; Use M6 after TB or 00 (i.e U0511XX-TB- M6)
Rear IV Pole (Right)	Add PC as illustrated U0521XX-00-00- PC
Foley Bag Holder (Right)	Add R0 as illustrated U0521XX-00-00- R0
Rear IV Pole and Foley Bag (Right)	Add RB as illustrated U0521XX-00-00- RB
Urethane Arm	Add UB as illustrated U0521XX-00-00-00- UB
<i>Left & Right is determined - as if sitting in the chair</i>	

For additional options please see: [**HM-HT OPTIONS PAGE**](#)

[**IV POLE OPTIONS PAGE**](#)

[**O2 HOLDER OPTIONS PAGE**](#)

[**SIDE TABLE OPTIONS PAGE**](#)

Rev. 01 – Initial Release.

NOTE: Contents of this document are subject to change without notice. Please refer to our website to be sure you are utilizing the most current Rev.

Page | 13
Copyright 2020 Winco Mfg., LLC

PL-5P-6P REV 01
Effective: 02/26/21

